

“凱石” 牙科植體系統

“Keystone” Dental Implant Systems

衛署醫器輸字第 012556 號

使用前請務必詳閱原廠之使用說明書並遵照指示使用。

產品敘述：

本產品包括了一個多樣化由鈦金屬經精密機械加工製造之植體；牙科植體是使用外科手術方式，將人工牙根植入上或下顎骨以取代自然牙根，作為日後復舊的穩固基礎。

可吸收介質噴覆表面處理(=RBM)的優點：

Keystone 是具加強骨整合之可吸收介質噴覆表面處理(=RBM)技術。

從 1997 年起 **Keystone** 的 RBM，一個具有表面粗糙的鈦金屬表面，一直在臨床上被使用。

Keystone RBM 表面處理技術，本質且充分地，可增強骨頭-to-植體的接觸和機械性骨頭的鎖入程度，比傳統機械表面有近 **250%**的粗糙度。經由磷酸鈣陶瓷媒介使用於粗糙面的處理上，可確保產品安全與生物適應性的優越性。

適應症：

Keystone 的植體是使用在部份及完全無牙的上下顎骨，並連結單一及多重復舊裝置，其中包括黏著式固持，螺絲固持或覆蓋式假牙復舊補綴體，於末端或中間接合支撐台，提供架橋固定。

使用方向：

植牙程序是在無菌的情況下，使用特別設計並經滅菌過的外科手術器械進行之，我們建議必須使用附加內或外供水(無菌水)的電動手術鑽研系統進行鑽研程序、特別鑽研順序以放置植體必須被小心審慎遵守，當使用埋頭孔鑽、導向鑽、深度鑽或結束鑽頭時，必須要有前後 In-Out 的動作，另外可使用手術指導手冊、深度探針及平行插針等協助放置植體。

螺紋植體：使用棘輪扳手(surgical ratchet)或電動手機(handpiece adaptor)將植體螺旋旋入預備放置植體處，持續將植體依順時鐘方向旋入植入處，直到完全進入固定，然後移除放置頭(placement head)，同時放入覆蓋螺絲，最後並縫合兩側皮瓣組織。

圓柱狀植體：押入(press)或攻入(tap)植體，最後並縫合兩側皮瓣組織。

禁忌症：

- 一般的禁忌症關係到外科時，要先觀察選擇過。
- 可能的禁忌症：慢性出血問題、心理損傷、代謝性變化骨頭或結締組織疾病、使用類固醇、心血管疾病、抽菸者、無法控制的糖尿病患、使用化療藥劑、慢性腎臟疾病、口腔衛生習慣不好者、磨牙、酒精中毒。
- 暫時的禁忌症：全身性感染、局部口腔及呼吸道感染。
- 解剖學及病理學禁忌症：
 1. 不足夠的齒槽骨高度和寬度（從頰與舌兩側至少須有大於 1mm 的骨量去圍繞植體）。
 2. 不足夠的骨高度，剛剛好大小的植體，將會侵害（侵蝕）上顎牙根管。
 3. 惡性疾病。

警告：

- 不適當的技術會造成植體植入的失敗及骨頭流失。
- Keystone 的牙科植體必須依適應症正確使用，不可任意更改規格與使用方法。
- 不建議使用使用電鍍外科手術器械或雷射，用於具有金屬性的植體及支撐台，這將會具有風險及電熱的傳導。
- 植體的移動、骨頭流失或慢性感染將導致植體植入的失敗問題。
- 若因病患體液污染了植體，就不能再用於其他病人身上。

預防：

- 植牙外科手術技巧，需要特殊與複雜的技術；因此正規牙科植體植入訓練是必需的。
- **重要：**
 1. 決定擺放植體的解剖適當位置與適量的骨頭。
 2. 治療前適當的 X 光片、觸診及植入位置的目視檢查是必要的

副作用：

- 可能造成的併發症：感染發炎、骨頭流失、病人的不適、植體的移動、植體的斷裂、局部軟組織惡化、不適當的植體放置與安排。

滅菌方式：

1. Gamma 滅菌
2. 不可重複滅菌
3. 破損包裝、效期超過或被打開污染時請勿使用

產品型號：見附錄

製造廠名稱：Keystone Dental, Inc.

製造廠地址：144 Middlesex Turnpike Burlington, MA 01803, U.S.A.

藥商名稱：杏昌生技股份有限公司

藥商地址：新北市三重區重新路五段 609 巷 2 號 8 樓之 2

STAGE-1 RBM Single Stage Implants		STAGE-1 RBM 單一階段人工植體*
新方單		
Description		品名
3.3mm RDS with 1.8mm Collar	*Includes cover screw & pre-mounted color-coded placement head	3.3mm 帶規直徑與 1.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
RSR3308-2K	3.3mm dia. x 8mm	3.3mm 直徑 x 8mm
RSR3310-2K	3.3mm dia. x 10mm	3.3mm 直徑 x 10mm
RSR3312-2K	3.3mm dia. x 12mm	3.3mm 直徑 x 12mm
RSR3314-2K	3.3mm dia. x 14mm	3.3mm 直徑 x 14mm
3.3mm RDS with 2.8mm Collar		3.3mm 帶規直徑與 2.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSR3308-3K	3.3mm dia. x 8mm	3.3mm 直徑 x 8mm
RSR3310-3K	3.3mm dia. x 10mm	3.3mm 直徑 x 10mm
RSR3312-3K	3.3mm dia. x 12mm	3.3mm 直徑 x 12mm
RSR3314-3K	3.3mm dia. x 14mm	3.3mm 帶規直徑與 1.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
4.1mm RDS with 1.8mm Collar		4.1mm 帶規直徑與 1.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSR4108-2K	4.1mm dia. x 8mm	4.1mm 直徑 x 8mm
RSR4110-2K	4.1mm dia. x 10mm	4.1mm 直徑 x 10mm
RSR4112-2K	4.1mm dia. x 12mm	4.1mm 直徑 x 12mm
RSR4114-2K	4.1mm dia. x 14mm	4.1mm 直徑 x 14mm
RSR4116-2K	4.1mm dia. x 16mm	4.1mm 直徑 x 16mm
4.1mm RDS with 2.8mm Collar		4.1mm 帶規直徑與 2.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSR4108-3K	4.1mm dia. x 8mm	4.1mm 直徑 x 8mm
RSR4110-3K	4.1mm dia. x 10mm	4.1mm 直徑 x 10mm
RSR4112-3K	4.1mm dia. x 12mm	4.1mm 直徑 x 12mm
RSR4114-3K	4.1mm dia. x 14mm	4.1mm 直徑 x 14mm
RSR4116-3K	4.1mm dia. x 16mm	4.1mm 直徑 x 16mm
4.8mm RDS with 1.8mm Collar		4.8mm 帶規直徑與 1.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSR4808-2K	4.8mm dia. x 8mm	4.8mm 直徑 x 8mm
RSR4810-2K	4.8mm dia. x 10mm	4.8mm 直徑 x 10mm
RSR4812-2K	4.8mm dia. x 12mm	4.8mm 直徑 x 12mm
RSR4814-2K	4.8mm dia. x 14mm	4.8mm 直徑 x 14mm
RSR4816-2K	4.8mm dia. x 16mm	4.8mm 直徑 x 16mm
4.8mm RDS with 2.8mm Collar		4.8mm 帶規直徑與 2.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSR4808-3K	4.8mm dia. x 8mm	4.8mm 直徑 x 8mm
RSR4810-3K	4.8mm dia. x 10mm	4.8mm 直徑 x 10mm
RSR4812-3K	4.8mm dia. x 12mm	4.8mm 直徑 x 12mm
RSR4814-3K	4.8mm dia. x 14mm	4.8mm 直徑 x 14mm
RSR4816-3K	4.8mm dia. x 16mm	4.8mm 直徑 x 16mm
4.8mm WDS with 1.8mm Collar		4.8mm 寬直徑與 1.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽
*Includes cover screw & pre-mounted color-coded placement head		
RSW4808-2K	4.8mm dia. x 8mm	4.8mm 直徑 x 8mm
RSW4810-2K	4.8mm dia. x 10mm	4.8mm 直徑 x 10mm
RSW4812-2K	4.8mm dia. x 12mm	4.8mm 直徑 x 12mm
RSW4814-2K	4.8mm dia. x 14mm	4.8mm 直徑 x 14mm
STAGE-1 RBM Single Stage Implants (Continued)		STAGE-1 RBM 單一階段人工植體
Catalog #	Description	
產品編號	*Includes cover screw & pre-mounted color-coded placement head	4.8mm 寬直徑與 2.8mm 頸圈 包括外蓋鏤絲及顏色標明裝置帽

RSW4808-3K	4.8mm dia. x 8mm	4.8mm 直徑 x 8mm
RSW4810-3K	4.8mm dia. x 10mm	4.8mm 直徑 x 10mm
RSW4812-3K	4.8mm dia. x 12mm	4.8mm 直徑 x 12mm
RSW4814-3K	4.8mm dia. x 14mm	4.8mm 直徑 x 14mm
	5.5mm WDS with 1.8mm Collar	5.5mm 直徑與 1.8mm 頸圈
	*Includes cover screw & pre-mounted color-coded placement head	包括外蓋螺絲及顏色標明裝置帽
RSW5508-2K	5.5mm dia. x 8mm	5.5mm 直徑 x 8mm
RSW5510-2K	5.5mm dia. x 10mm	5.5mm 直徑 x 10mm
RSW5512-2K	5.5mm dia. x 12mm	5.5mm 直徑 x 12mm
RSW5514-2K	5.5mm dia. x 14mm	5.5mm 直徑 x 14mm
	5.5mm WDS with 2.8mm Collar	5.5mm 直徑與 2.8mm 頸圈
	*Includes cover screw & pre-mounted color-coded placement head	包括外蓋螺絲及顏色標明裝置帽
RSW5508-3K	5.5mm dia. x 8mm	5.5mm 直徑 x 8mm
RSW5510-3K	5.5mm dia. x 10mm	5.5mm 直徑 x 10mm
RSW5512-3K	5.5mm dia. x 12mm	5.5mm 直徑 x 12mm
RSW5514-3K	5.5mm dia. x 14mm	5.5mm 直徑 x 14mm
	6.3mm WDS with 1.8mm Collar	6.3mm 直徑與 1.8mm 頸圈
	*Includes cover screw & pre-mounted color-coded placement head	包括外蓋螺絲及顏色標明裝置帽
RSW6308-2K	6.3mm dia. x 8mm	6.3mm 直徑 x 8mm
RSW6310-2K	6.3mm dia. x 10mm	6.3mm 直徑 x 10mm
	Cover Screws	外蓋螺絲
Catalog #	Description	
	Cover Screws (RDS)	外蓋螺絲(常規直徑)
S1800-00K	0.5mm height-headless (RDS)	0.5mm 高-無頭式(常規直徑)
S1800-15K	1.5mm height (RDS)	1.5mm 高(常規直徑)
S1800-30K	3.0mm height (RDS)	3.0mm 高(常規直徑)
S1800-45K	4.5mm height (RDS)	4.5mm 高(常規直徑)
S1806-20K	2.0mm height (WDS)	2.0mm 高(寬直徑)
S1806-30K	3.0mm height (WDS)	3.0mm 高(寬直徑)
S1806-45K	4.5mm height (WDS)	4.5mm 高(寬直徑)
	Cement-on Crown (COC) Abutment System	黏著式牙冠用連接體系統
Catalog #	Description	
	COC QUICK-PACK (RDS)	黏著式牙冠用連接體快拆包(常規直徑)
S2442-40K	STAGE-1 RDS COC Quick Pack 4.0mm	STAGE-1 常規直徑黏著式牙冠用連接體快拆包 4.0mm
S2442-55K	STAGE-1 RDS COC Quick Pack 5.5mm	STAGE-1 常規直徑黏著式牙冠用連接體快拆包 5.5mm
S2442-70K	STAGE-1 RDS COC Quick Pack 7.0mm	STAGE-1 常規直徑黏著式牙冠用連接體快拆包 7.0mm
	COC QUICK-PACK (WDS)	黏著式牙冠用連接體快拆包(寬直徑)
S2507-40K	STAGE-1 WDS COC Quick Pack 4.0mm	STAGE-1 寬直徑黏著式牙冠用連接體快拆包 4.0mm
S2507-55K	STAGE-1 WDS COC Quick Pack 5.5mm	STAGE-1 寬直徑黏著式牙冠用連接體快拆包 5.5mm
S2507-70K	STAGE-1 WDS COC Quick Pack 7.0mm	STAGE-1 寬直徑黏著式牙冠用連接體快拆包 7.0mm
	COC Abutment Try-in Set (WDS)	黏著式牙冠用連接體試驗體(寬直徑)
	Try-in Set (WDS), 4.0, 5.5 and 7.0mm (sky blue, gold, silver)	試驗體(寬直徑), 4.0, 5.5 and 7.0mm (A, B, C)
S2503K		4.0mm 高替代物(常規直徑), 紫色
S2433-40-1K	4.0mm height Analog (RDS), violet	4.0mm 高替代物(常規直徑), 紫色
S2433-55-1K	5.5mm height Analog (RDS), green	5.5mm 高替代物(常規直徑), 綠色
S2433-70-1K	7.0mm height Analog (RDS), navy blue	7.0mm 高替代物(常規直徑), 藍色
	COC Abutment Analog (WDS)	黏著式牙冠用連接體替代物(寬直徑)
S2502-40-1K	4.0mm height Analog (WDS), sky blue	4.0mm 高替代物(寬直徑), 藍色
S2502-55-1K	5.5mm height Analog (WDS), gold	5.5mm 高替代物(寬直徑), 金色
S2502-70-1K	7.0mm height Analog (WDS), silver	7.0mm 高替代物(寬直徑), 銀色
	COC Abutment (RDS)	黏著式牙冠用連接體(常規直徑)
S2432-40-1K	4.0mm height COC (RDS), violet	4.0mm 高黏著式牙冠用連接體(常規直徑) 紫色
S2432-55-1K	5.5mm height COC (RDS), green	5.5mm 高黏著式牙冠用連接體(常規直徑) 綠色
S2432-70-1K	7.0mm height COC (RDS), navy blue	7.0mm 高黏著式牙冠用連接體(常規直徑) 藍色

	COC Abutment (WDS)		黏著式牙冠用連接體(寬直徑)
S2501-40-1K	4.0mm height COC (WDS), sky blue		4.0mm 高 黏著式牙冠用連接體(寬直徑), 藍色
S2501-55-1K	5.5mm height COC (WDS), gold		5.5mm 高 黏著式牙冠用連接體(寬直徑), 金色
S2501-70-1K	7.0mm height COC (WDS), silver		7.0mm 高 黏著式牙冠用連接體(寬直徑), 銀色
	COC Angled Abutment (RDS)		黏著式牙冠用具角度型連接體(常規直徑)
42001K	15° Angled Abutment		15° 連接體(常規直徑)
42002K	20° Angled Abutment		20° 連接體(常規直徑)
42000K	Angled Abutment Screw		具角度型連接體螺絲
S2435K	Custom Prep COC Abutment		黏著式牙冠用預先定製連接體
S2504K	Custom Prep COC Abutment (RDS)		黏著式牙冠用預先定製連接體(常規直徑)
S2505K	Custom Prep COC Abutment (WDS)		黏著式牙冠用預先定製連接體(寬直徑)
	Implant Analog (WDS)		人工植體替代物(寬直徑)
	Cement-on Crown (COC) Abutment System (Continued)		黏著式牙冠用連接體
	Description		
	COC Quick-Cap (RDS)		黏著式牙冠用連接體快速帽(常規直徑)
產品編號			
S2431-40K	4.0mm height Quick-Cap (RDS), violet, 4 pack		4.0mm 高 快速帽(常規直徑), A, 4包裝
S2431-55K	5.5mm height Quick-Cap (RDS), green, 4 pack		5.5mm 高 快速帽(常規直徑), B, 4包裝
S2431-70K	7.0mm height Quick-Cap (RDS), navy blue, 4 pack		7.0mm 高 快速帽(常規直徑), C, 4包裝
	COC Abutment Temporary Cap (RDS)		黏著式牙冠用連接體臨時帽(常規直徑)
S2439K	COC Abutment Temporary Cap (RDS)		黏著式牙冠用連接體臨時帽(常規直徑)
	COC Abutment Temporary Cap (WDS)		黏著式牙冠用連接體臨時帽(寬直徑)
S2506K	COC Abutment Temporary Cap (WDS)		黏著式牙冠用連接體臨時帽(寬直徑)
	COC Abutment Plastic Sleeve (RDS)		黏著式牙冠用連接體塑膠套筒(常規直徑)
S2437-00-1K	Plastic Sleeve, Locking (RDS), white		塑膠套筒, 固定型(常規直徑),
S2437-01-1K	Plastic Sleeve, Non-locking (RDS), white		塑膠套筒, 非固定型(常規直徑),
	COC Abutment Plastic Sleeve (WDS)		黏著式牙冠用連接體塑膠套筒(寬直徑)
S2438-00-1K	Plastic Sleeve, Locking (WDS), white		塑膠套筒, 固定型(寬直徑),
S2438-01-1K	Plastic Sleeve, Non-locking (WDS), white		塑膠套筒, 非固定型(寬直徑),
	COC Reamer TOOL		黏著式牙冠用擴孔器
	Indexed Abutment System		指引連接體系統
	Description		
產品編號			
	Indexed Abutment		指引連接體
S2400K	Abutment (RDS)		連接體(常規直徑)
42004K	Abutment (WDS)		連接體(寬直徑)
	Indexed Abutment Analog		指引連接體替代物
S2404K	Analog (RDS)		替代物(常規直徑)
42006K	Analog (WDS)		替代物(寬直徑)
	Indexed Abutment Gold Sleeve		指引連接體金屬套筒
S2405-00K	Gold Sleeve, Locking (RDS)		金屬套筒, 固定型(常規直徑)
S2405-01K	Gold Sleeve, Non-locking (RDS)		金屬套筒, 非固定型(常規直徑)
42007K	Gold Sleeve, Locking (WDS)		金屬套筒, 固定型(寬直徑)
42008K	Gold Sleeve, Non-locking (WDS)		金屬套筒, 非固定型(寬直徑)
	Indexed Abutment Plastic Sleeve		指引連接體塑膠套筒
S2406-00K	Plastic Sleeve, Locking (RDS)		塑膠套筒, 固定型(常規直徑)
S2406-01K	Plastic Sleeve, Non-locking (RDS)		塑膠套筒, 非固定型(常規直徑)
42009K	Plastic Sleeve, Locking (WDS)		塑膠套筒, 固定型(寬直徑)
42010K	Plastic Sleeve, Non-locking (WDS)		塑膠套筒, 非固定型(寬直徑)
	Indexed Abutment Temporary Sleeve		指引連接體臨時套筒

S2407-00K	Temporary Sleeve, Locking (RDS)	臨時套筒，固定型 (常規直徑)
S2407-01K	Temporary Sleeve, Non-locking (RDS)	臨時套筒，非固定型 (常規直徑)
42011K	Temporary Sleeve, Locking (WDS)	臨時套筒，固定型 (寬直徑)
42012K	Temporary Sleeve, Non-locking (WDS)	臨時套筒，非固定型 (寬直徑)
S2402K	Indexed Abutment Impression Post Impression Post (RDS)	索引連接體印模 印模 (常規直徑)
42013K	Impression Post (WDS)	印模 (寬直徑)
S2408K	Indexed Abutment Screw, Titanium For RDS STAGE-1 Screw (RDS)	索引連接體螺絲 螺絲 (常規直徑)
42014K	Screw (WDS)	螺絲 (寬直徑)
O-Ring Abutment System	O-Ring Abutment System Description	O-Ring 連接體系統 ØRing 連接體
S2417-0K	0mm height (RDS)	0mm 高 (常規直徑)
S2417-2K	2mm height (RDS)	2mm 高 (常規直徑)
L2805-01K	O-Ring Abutment Accessories Abutment Analog	O-Ring 連接體配件 連接體替代物
L2810-01K	Keeper, Titanium	固著物，鈦合金
Surgical Instrumentation	Surgical Instrumentation Description	手術用儀器
Master Surgical Kit for STAGE-1 Single Stage Implants	Master Surgical Kit for STAGE-1 Single Stage Implants S9101-1K S9101-2K	STAGE-1 單一階段人工植體專業手術器械組 專業手術套件基本版，內沖水鑽頭 專業手術套件基本版，非內沖水鑽頭
S1801-01K	Master Surgical Tray for STAGE-1 Single Stage Implants (without instruments) Master Kit Surgical Tray only	STAGE-1 單一階段人工植體專業手術牙托 (未包含儀器組)
Surgical Instrumentation (Continued)	Surgical Instrumentation (Continued) Catalog # Description	專業套件手術牙托 手術用儀器
產品編號	Twist Drills	旋轉鑽頭
S1101-20-10K	2.0 x 10mm Twist Drill, Irrigated	2.0 x 10mm 旋轉鑽頭，內沖水
S1101-20-16K	2.0 x 16mm Twist Drill, Irrigated	2.0 x 16mm 旋轉鑽頭，非內沖水
S1101-28-10K	2.8 x 10mm Twist Drill, Irrigated	2.8 x 10mm 旋轉鑽頭，內沖水
S1101-28-16K	2.8 x 16mm Twist Drill, Irrigated	2.8 x 16mm 旋轉鑽頭，非內沖水
S1100-28-16K	2.8 x 16mm Twist Drill, Non-Irrigated	2.8 x 16mm 旋轉鑽頭，非內沖水
S1101-35-10K	3.5 x 10mm Twist Drill, Irrigated	3.5 x 10mm 旋轉鑽頭，內沖水
S1101-35-16K	3.5 x 16mm Twist Drill, Irrigated	3.5 x 16mm 旋轉鑽頭，內沖水
S1100-35-10K	3.5 x 10mm Twist Drill, Non-Irrigated	3.5 x 10mm 旋轉鑽頭，非內沖水
S1100-35-16K	3.5 x 16mm Twist Drill, Non-Irrigated	3.5 x 16mm 旋轉鑽頭，非內沖水
S1101-42-10K	4.2 x 10mm Twist Drill, Irrigated	4.2 x 10mm 旋轉鑽頭，內沖水
S1101-42-16K	4.2 x 16mm Twist Drill, Irrigated	4.2 x 16mm 旋轉鑽頭，內沖水
S1100-42-10K	4.2 x 10mm Twist Drill, Non-Irrigated	4.2 x 10mm 旋轉鑽頭，非內沖水
S1100-42-16K	4.2 x 16mm Twist Drill, Non-Irrigated	4.2 x 16mm 旋轉鑽頭，非內沖水
S1101-48-10K	4.8 x 10mm Twist Drill, Irrigated	4.8 x 10mm 旋轉鑽頭，內沖水
S1101-48-16K	4.8 x 16mm Twist Drill, Irrigated	4.8 x 16mm 旋轉鑽頭，內沖水

S1100-48-10K	4.8 x 10mm Twist Drill, Non-irrigated	4.8 x 10mm 旋轉鑽頭, 非內沖水
S1100-48-16K	4.8 x 16mm Twist Drill, Non-irrigated	4.8 x 16mm 旋轉鑽頭, 非內沖水
S1101-55-10K	5.5 x 10mm Twist Drill, Irrigated	5.5 x 10mm 旋轉鑽頭, 內沖水
S1100-55-10K	5.5 x 10mm Twist Drill, Non-irrigated	5.5 x 10mm 旋轉鑽頭, 非內沖水
Surgical Taps		
S1500-33-10K	3.3 x 10mm Surgical Tap, Irrigated	3.3 x 10mm 手術用攻牙鑽頭, 內沖水
S1500-33-16K	3.3 x 16mm Surgical Tap, Irrigated	3.3 x 16mm 手術用攻牙鑽頭, 內沖水
S1501-41-10K	4.1 x 10mm Surgical Tap	4.1 x 10mm 手術用攻牙鑽頭
S1501-41-16K	4.1 x 16mm Surgical Tap	4.1 x 16mm 手術用攻牙鑽頭
S1501-48-10K	4.8 x 10mm Surgical Tap	4.8 x 10mm 手術用攻牙鑽頭
S1501-48-16K	4.8 x 16mm Surgical Tap	4.8 x 16mm 手術用攻牙鑽頭
S1500-55-10K	5.5 x 10mm Surgical Tap, Irrigated	5.5 x 10mm 手術用攻牙鑽頭, 內沖水
S1500-55-16K	5.5 x 16mm Surgical Tap, Irrigated	5.5 x 16mm 手術用攻牙鑽頭, 內沖水
S1500-63-10K	6.3 x 10mm Surgical Tap, Irrigated	6.3 x 10mm 手術用攻牙鑽頭, 內沖水
Conversion Adapter & Surgical Ratchet		
L1702-01K	Surgical Ratchet	手術用齒輪
Ratchet Adapters for STAGE-1 Placement Heads		
S1701-01K	Ratchet Adapter, Short	齒輪接頭, 短型
S1701-02K	Ratchet Adapter, Medium	齒輪接頭, 中型
S1701-03K	Ratchet Adapter, Long	齒輪接頭, 長型
L1806-03K	Replacement O-Rings for Ratchet Adapters (4pk)	O-Rings取代裝置齒輪接頭(4包裝)
IS1112K	Conversion Handle for Accu-Torque Drivers and Ratchet Adapters, Swivel Head	Accu-Torque 旋轉起子轉向握柄及齒輪接頭, 旋轉頭
STAGE-1 Placement Heads		
S1601-01K	Placement Head (RDS)	裝置頭(常規直徑)
S1805K	Placement Head (WDS)	裝置頭(寬直徑)
Stabilizing Wrench for STAGE-1 Placement Heads		
S1700K	Stabilizing Wrench	穩定性扳鉗
Handpiece Adapter for STAGE-1 Placement Heads		
S1400-01K	Handpiece Adapter, Short	手機接頭, 短型
S1400-02K	Handpiece Adapter, Medium	手機接頭, 中型
S1400-03K	Handpiece Adapter, Long	手機接頭, 長型
Surgical Hand Driver		
S1402-01K	Surgical Hand Driver	手術用手用旋轉起子
Locking Drill Extender		
L1404-01K	Locking Drill Extender	固定式鑽頭延長器
Surgical Depth Probe		
S1802K	Surgical Depth Probe	手術用深度探針
Depth Gauges		
S1803-28K	2.8mm Depth Gauge	深度測量尺
S1803-33K	3.5mm Depth Gauge	3.5mm 深度測量尺
S1803-42K	4.2mm Depth Gauge	4.2mm 深度測量尺
S1803-48K	4.8mm Depth Gauge	4.8mm 深度測量尺
S1803-55K	5.5mm Depth Gauge	5.5mm 深度測量尺
Drivers		
.048" Hex Drivers		
R9121K	Swivel Head, Short (8mm shaft/19mm total length)	旋轉頭, 短型 (8mm 握柄/19mm 總長)

R9122K	Swivel Head, Long (13mm shaft/24mm total length)	旋轉頭, 長型 (13mm 握柄/24mm 總長)
L1400-48-23K	Latch-Type, Short (23mm total length)	Latch-Type, 短型 (23mm 總長)
L1400-48-28K	Latch-Type, Long (28mm total length)	Latch-Type, 長型 (28mm 總長)
產品編號	COC Abutment Drivers	黏著式牙冠用連接體旋轉起子
S2420-1K	Latch-Type (RDS) (23mm total length)	Latch-Type (常規直徑) (23mm 總長)
S2428K	Latch-Type (WDS) (23mm total length)	Latch-Type (寬直徑) (23mm 總長)
	.062 " Hex Drivers	.062六角旋轉起子
L2100-62-19K	Swivel Head, Short (8mm shaft/19mm total length)	旋轉頭, 短型 (8mm 握柄/19mm 總長)
L2100-62-24K	Swivel Head, Long (13mm shaft/24mm total length)	旋轉頭, 長型 (13mm 握柄/24mm 總長)
	Overdenture Abutment Drivers	覆蓋式連接體旋轉起子
L1404-96-20K	Latch-Type (23mm total length)	Latch-Type (23mm 總長)

Restore RBM 螺旋植體		Description
HA Coated Threaded Implants		氫氣鍍灰土灘蓋螺旋植體
Cover Screws		
	Description: Implant Size: Qty	
Cover Screws for External Hex Implants		
L1809-34-01K	Straight Cover Screw, SD-1	外蓋螺絲適用於六角植體
L1809-40-01K	Straight Cover Screw, RD-1	平直型外蓋螺絲小直徑
L1809-50-01K	Straight Cover Screw, WD-1	平直型外蓋螺絲大直徑
L1810-40-01K	Headless Cover Screw, RD-1	平直型外蓋螺絲實直徑
L1810-50-01K	Headless Cover Screw, WD-1	無頭型外蓋螺絲實直徑
Aesthetic Contour (AC) Healing Abutments		
	Description: Implant Size: Qty	
Implant Size/Flare Dia./Cuff Ht./Qty		
Implant Hex SD Healing Abutments for External Hex Implants		
R9633-34-02K	SD/3mm non-flared (straight)/2mm cuff;1	圓錐號碼/外徑直徑/袖口
R9633-34-04K	SD/3mm non-flared (straight)/4mm cuff;1	104° 六角小直徑蓋合蓋螺絲適用於六角形植體
R9633-34-06K	SD/3mm non-flared (straight)/6mm cuff;1	小直徑 3.0mm 無外徑平直型2mm
Implant Hex WD Healing Abutments for External Hex Implants		
R9633-50-02K	WD/5.0mm flare (straight)/2mm cuff;1	小直徑 3.0mm 無外徑平直型6mm
R9633-50-04K	WD/5.0mm flare (straight)/4mm cuff;1	寬扁直徑 5.0mm/外徑平直型2mm
R9633-50-06K	WD/5.0mm flare (straight)/6mm cuff;1	寬扁直徑 5.0mm/外徑平直型6mm
Implant Impression Components		
R9649-34-00K	SD/3.5mm non-flared (straight), 104° Hex)	小直徑 3.5mm 無外徑平直型
R9649-40-00K	RD/4.1mm non-flared (straight), 104° Hex)	小直徑 4.1mm 無外徑平直型
R9649-50-00K	WD/5.0mm non-flared (straight), 104° Hex)	特別直徑 5.0mm 無外徑直徑
Non-flared UCLA Impression Posts		
R9693-34-01K	SD Non-flared 104° Hex)	小直徑 無六角印模
R9693-40-01K	RD Non-flared 104° Hex)	特別直徑 無六角印模
R9693-50-01K	WD Non-flared 104° Hex)	寬直徑 無六角印模
Implant Analogs		
R9893-34K	SD Implant Analog	小直徑植體替代物
R9891-40K	RD Implant Analog	特別直徑植體替代物
R9892-40K	RD Implant Analog -3-Pack	特別直徑植體替代物
R9893-50K	WD Implant Analog	寬直徑植體替代物
Cement-on Crown (COC) Abutment System		
	Description	粘著式牙冠用連接物系統
Catalog #	Description	
Straight Locking Cement-on Crown (COC) Abutments		
R9290-34-02K	SD 2mm Cuff	直徑鎖式粘著式牙冠用連接物
R9290-34-04K	SD 4mm Cuff	SD 2mm Cuff
R9290-40-02K	RD 2mm Cuff	SD 4mm Cuff
R9290-40-04K	RD 4mm Cuff	RD 2mm Cuff
R9290-50-02K	WD 2mm Cuff	WD 2mm Cuff
R9290-50-04K	WD 4mm Cuff	WD 4mm Cuff
Straight Non-Locking Cement-on Crown (COC) Abutments		
R9292-42-02K	SD 104° Hex)	SD 104° Hex)
R9292-42-04K	RD 104° Hex)	RD 104° Hex)
R9292-50-02K	WD 104° Hex)	WD 104° Hex)
Abutment Screws for Cement-on Crown (COC) Abutments		
R9203-34-48K	SD Titanium 104° Hex)	螺絲頭式粘著式牙冠用連接物
R9202-40-48K	RD Titanium 104° Hex)	SD Titanium 104° Hex)
R9203-50-48K	WD Titanium 104° Hex)	RD Titanium 104° Hex)
UCLA Abutment System		
	Description	
Abutment Screws for UCLA Abutments		
R9203-34-48K	SD Titanium 104° Hex)	螺絲頭式螺絲適用於六角牙橋
R9202-40-48K	SD Titanium 104° Hex)	SD Titanium 104° Hex)
R9203-50-48K	WD Titanium 104° Hex)	SD Titanium 104° Hex)
Conical Abutment System		
	Description	
Conical Abutments		
L2300-33-01K	SD/1mm Cuff, with screw	圓錐連接體
L2300-33-02K	SD/2mm Cuff, with screw	SD/1mm Cuff, with screw
L2300-33-03K	SD/3mm Cuff, with screw	SD/2mm Cuff, with screw
R9210-40-01K	RD/1mm Cuff, with screw	SD/3mm Cuff, with screw
R9210-40-02K	RD/2mm Cuff, with screw	RD/1mm Cuff, with screw
R9210-40-03K	RD/3mm Cuff, with screw	RD/2mm Cuff, with screw
R9210-50-01K	WD/1mm Cuff, with screw	RD/3mm Cuff, with screw
		WD/1mm Cuff, with screw

R9210-50-02K	WD/2mm Cuff, with screw	WD/2mm Cuff, with screw
R9210-50-03K	WD/3mm Cuff, with screw	WD/3mm Cuff, with screw
Healing Caps for Conical Abutments		
R9212-40K	SD Healing Cap (048° Hex)	SD Healing Cap (048° Hex)
R9212-40K	RD Healing Cap (048° Hex)	RD Healing Cap (048° Hex)
R9212-55-48K	WD Healing Cap (048° Hex)	WD Healing Cap (048° Hex)
Impression Posts for Conical Abutments		
R9330-40-01K	SD/Non-hexed/1-piece	SD/Non-hexed/1-piece
R9330-40-01K	RD/Non-hexed/1-piece	RD/Non-hexed/1-piece
Analogs for Conical Abutments		
R9945-40K	SD Conical Abutment Analog	SD Conical Abutment Analog
R9945-40K	RD Conical Abutment Analog	RD Conical Abutment Analog
R9945-55K	WD Conical Abutment Analog	WD Conical Abutment Analog
Plastic Sleeves for Conical Abutments		
R9211-40-01K	SD Plastic Sleeve, Non-hexed	SD Plastic Sleeve, Non-hexed
R9211-40-01K	RD Plastic Sleeve, Non-hexed	RD Plastic Sleeve, Non-hexed
R9211-55-01K	WD Plastic Sleeve, Non-hexed	WD Plastic Sleeve, Non-hexed
Temporary Sleeves for Conical Abutments		
R9214-40-01K	SD Temporary Sleeve, Non-hexed	SD Temporary Sleeve, Non-hexed
R9214-40-01K	RD Temporary Sleeve, Non-hexed	RD Temporary Sleeve, Non-hexed
L2700-50-01K	WD Temporary Sleeve, Non-hexed	WD Temporary Sleeve, Non-hexed
Fastening Screws for Conical Abutments		
R9200-40-48K	SD Titanium (048° Hex)	SD Titanium (048° Hex)
R9201-40-48K	SD Gold (048° Hex)	SD Gold (048° Hex)
R9200-40-48K	RD Titanium (048° Hex)	RD Titanium (048° Hex)
R9201-40-48K	RD Gold (048° Hex)	RD Gold (048° Hex)
R9201-55-48K	WD Titanium (048° Hex)	WD Titanium (048° Hex)
Standard Abutment System		
標準連接系統		
Standard Abutments		
R9086-40-03K	RD 3mm Cuff	RD 3mm Cuff
R9086-40-04K	RD 4mm Cuff	RD 4mm Cuff
R9086-40-05K	RD 5mm Cuff	RD 5mm Cuff
Healing Caps for Standard Abutments		
R9294-40K	RD Healing Cap (048° Hex)	RD Healing Cap (048° Hex)
R9886-40K	RD Standard Abutment Analog	RD Standard Abutment Analog
O-Ring Abutment System		
全口假牙連接系統		
O-Ring Abutments		
L2304-42-02K	SD 2mm Cuff	SD 2mm Cuff
L2304-42-03K	SD 3mm Cuff	SD 3mm Cuff
L2304-42-04K	SD 4mm Cuff	SD 4mm Cuff
L2402-50-02K	RD 2mm Cuff	RD 2mm Cuff
L2402-50-03K	RD 3mm Cuff	RD 3mm Cuff
L2402-50-04K	RD 4mm Cuff	RD 4mm Cuff
L2505-60-02K	WD 2mm Cuff	WD 2mm Cuff
L2505-60-03K	WD 3mm Cuff	WD 3mm Cuff
O-Ring Abutment Accessories		
L2805-01K	Abutment Analog	Abutment Analog
L2810-01K	Keeper, Titanium	Keeper, Titanium
Threaded Implant Surgical Kits		
螺旋紋植體手術盒		
Basic Master Surgical Kit for Threaded Implants - Non-Irrigated*		
*Refer to Surgical Product Catalog for full listing of included items.		
Drills & Taps		
牙鑽		
Round Burs		
L1405-23-01K	Round Marking Bur, Non-irrigated, 2.3mm	扭鑽
Twist Drills - 3mm SD Threaded Implant Finishing Drills		
L1101-28-10K	2.8 x 10mm Non-irrigated Twist Drill	2.8 x 10mm Non-irrigated Twist Drill
L1101-28-15K	2.8 x 15mm Non-irrigated Twist Drill	2.8 x 15mm Non-irrigated Twist Drill
Pilot Drills		
L1300-20-30K	2.0/3.0mm Pilot Drill, Irrigated	2.0/3.0mm Pilot Drill, Irrigated
L1301-20-30K	2.0/3.0mm Pilot Drill, Non-irrigated	2.0/3.0mm Pilot Drill, Non-irrigated
L1300-30-42K	3.0/4.2mm Pilot Drill, Irrigated	3.0/4.2mm Pilot Drill, Irrigated
L1300-34-42K	3.4/4.2mm Pilot Drill, Irrigated	3.4/4.2mm Pilot Drill, Irrigated
L1300-42-50K	4.2/5.0mm Pilot Drill, Irrigated	4.2/5.0mm Pilot Drill, Irrigated
L1300-42-52K	4.2/5.2mm Pilot Drill, Irrigated	4.2/5.2mm Pilot Drill, Irrigated
Twist Surgical Taps (LATCH-TYPE) - 3.3mm SD Taps for Threaded Implants		
L1302-33-13K	3.3 x 13mm Surgical Tap	攻牙鑽頭
L1302-33-18K	3.3 x 18mm Surgical Tap	3.3 x 18mm Surgical Tap

	Bone Profiler System for External Hex Implants (SD)			
L141-01K	Guide Pin for SD Bone Profiler (2 pack)	骨子溝除雜	Guide Pin for SD Bone Profiler (2 pack)	p39
	Bone Profiler System for External Hex Implants (RD)			
L1412-01K	Guide Pin for RD Bone Profiler (2 pack)	骨子溝除雜	Guide Pin for RD Bone Profiler (2 pack)	p39
	Bone Profiler System for External Hex Implants (WD)			
L1413-01K	Guide Pin for WD Bone Profiler (2 pack)	骨子溝除雜	Guide Pin for WD Bone Profiler (2 pack)	p39
	SD Narrow Profile (NP) Placement Heads for Threaded Implants			
R9192-38-08K	SD 5.5mm Placement Head, .048" Hex	骨子溝除雜	SD 5.5mm Placement Head, .048" Hex	p40
R9192-38-19K	SD 15mm Placement Head, .048" Hex	骨子溝除雜	SD 15mm Placement Head, .048" Hex	p40
	RD Narrow Profile (NP) Placement Heads for Threaded Implants			
R9192-40-08K	RD 5.5mm Placement Head, .048" Hex	骨子溝除雜	RD 5.5mm Placement Head, .048" Hex	p40
R9192-40-15K	RD 15mm Placement Head, .048" Hex	骨子溝除雜	RD 15mm Placement Head, .048" Hex	p40
	WD Narrow Profile (NP) Placement Heads for Threaded Implants			
R9192-50-15K	WD 15mm Placement Head, .048" Hex		Narrow Profile (NP) Placement Heads for WD 5.0, 5.5 & 6.0mm Hex Implants / 15mm Length, .048mm Hex Narrow Profile (NP) Placement H	p40
	Accessories for Placement Heads (includes one each titanium sleeve for SD, RD, and WD implants)			
L1813-01K	Placement Head Assembly Block		Placement Head Assembly Block With Titanium Sleeves	p40
	Open End Wrench			

	Stabilizing Wrench	Surgical and Restorative Instrumentation:
L1709-01K	Stabilizing Wrench	Stabilizing Wrench For Narrow Profile (NP) Placement Heads / Stabilizing Wrench
	Surgical Ratchet and Extenders	
L1702-01K	Surgical Ratchet	Surgical Ratchet / Surgical Ratchet
L1806-04K	Ratchet Adapter	Ratchet Adapter For NP Placement Heads / 8mm Length Ratchet Adapter
L1806-05K	Ratchet Adapter	Ratchet Adapter For NP Placement Heads and Surgical Taps / 12mm Length Ratchet Adapter
L1806-03K	Replacement O-Ring for Ratchet Adapter (4pk)	Replacement O-Rings, 4 Pack For use with Ratchet Adapters / Replacement O-Rings, 4 Pack
	Square Drivers	
L2100-85-19K	Square Head, Short (8mm shaft/19mm total length)	Square Swivel Head Driver, Short / 8mm Shaft, 19mm Length, Square Swivel Head Driver, Short
L2100-85-22K	Square Head, Long (13mm shaft/22mm total length)	Square Swivel Head Driver, Long / 13mm Shaft, 22mm Length, Square Swivel Head Driver, Long
L1400-85-28K	Latch Type, Short (28mm total length)	Square Latch Type Driver, Short For Gold Alignment Screws / 23mm Length Square Latch Type Driver, Short
L1400-85-28K	Latch Type, Long (28mm total length)	Square Latch Type Driver, Long For Gold Alignment Screws / 28mm Length Square Latch Type Driver, Long
ISS1118-1K	Driver for Accu-Torque Wrench Short (8mm shaft)	Square Driver, Short For Accu-Torque Wrench / 8mm Shaft Square Driver, Short
ISS1118K	Driver for Accu-Torque Wrench Long (15mm shaft)	Square Driver, Long For Accu-Torque Wrench / 15mm Shaft Square Driver, Long
	Standard/Conical Alignment Drivers	標準 / 圓錐 Drivers
ISS1103K	Driver for Accu-Torque Wrench (8mm shaft)	Driver for Accu-Torque Wrench (8mm shaft)
	Overerture Alignment Drivers	
L1404-96-20K	Latch Type (23mm total length)	Latch-Type (23mm total length)
	Lab Tools & Other Products	
	Description	
	Component Pack	配件
L1812-01K	Component Pack	Component Pack
	Reamers	配件
L2802-01K	Reamer Handle, Swivel Head	Reamer Handle, Swivel Head
L2800-01K	Reamer, SD UCLA	Reamer, SD UCLA
L2800-02K	Reamer, RD UCLA	Reamer, RD UCLA
L2800-03K	Reamer, WD UCLA	Reamer, WD UCLA
	Waxing Pins	配件
R9394-34K	SD UCLA Waxing (1048° Hex)	SD UCLA Waxing (1048° Hex)
R9995-50-48K	WD UCLA Waxing (1048° Hex)	WD UCLA Waxing (1048° Hex)
	Header Bar and Clip System	附件
ISS1858BK	Header Castable Bar	Header Castable Bar
ISS1861PK	Header Processing Clips, 6 pk	Header Processing Clips, 6 pk
ISS1861HK	Header Final Clips, 6 pk	Header Final Clips, 6 pk
ISS922HK	Header Clip Metal Housings, 6 pk	Header Clip Metal Housings, 6 pk
	Hand Adapter for NP Placement Head	配件
L1406-02K	Hand Adapter for NP Placement Head	Hand Adapter for NP Placement Head
	Handpiece Placement Adapter	手機
L1406-01K	Handpiece Adapter (NP Placement Heads)	Handpiece Adapter (NP Placement Heads)
	Surgical Hand Driver	手機
L1414-01K	Surgical Hand Driver	Surgical Hand Driver
	Locking Drill Extender	Locking Drill Extender
L1404-01K	Locking Drill Extender	Locking Drill Extender
	Surgical Instrumentation (cont'd)	Surgical Instrumentation (cont'd)
	Irrigation Port Cleaner	配件
L1800-22-02K	Irrigation Port Cleaner	Irrigation Port Cleaner
	Forceps	鑷子
	Tissue Measuring Gauge	量尺
IS280-40K	Tissue Measuring Gauge (for RD implants)	Tissue Measuring Gauge (for RD implants)
	Tissue Punches - Disposable & Reusable	Tissue Punches - Disposable & Reusable
ISS140K	Tissue Punch, 3.7mm, Disposable	Tissue Punch, 3.7mm, Disposable
ISS141K	Tissue Punch, 4.5mm, Disposable	Tissue Punch, 4.5mm, Disposable
	Parallel Pins	直針
L1806-20-00K	Straight, 2.0/3.0mm, 4 pk	Straight, 2.0/3.0mm, 4 pk
L1806-20-15K	15 Degree, 2.0/3.0mm, 2 pk	15 Degree, 2.0/3.0mm, 2 pk
L1806-20-25K	25 Degree, 2.0/3.0mm, 2 pk	25 Degree, 2.0/3.0mm, 2 pk
	Gold Bar Cylinder	配件
R2402-01K	RD Gold Bar Cylinder, Non-Locking	RD Gold Bar Cylinder, Non-Locking
	Internal Thread Taps for Implants	攻牙鑽
L1700-27K	SD Internal Thread Tap	SD Internal Thread Tap
L1700-20K	RD Internal Thread Tap	RD Internal Thread Tap
L1700-25K	WD Internal Thread Tap	WD Internal Thread Tap

Prima


	Description	
Catalog #		品名
	PrimaConnex Cover Screws Primaconnex外蓋螺絲	
	Description	
	PrimaConnex Cover Screws	
15300K	SD, 0.5mm Height	小直徑, 0.5mm
15306K	RD, 0.5mm Height	常規 0.5mm
15312K	WD, 0.5mm Height	寬 0.5mm
	Implant Analogs 角度替代物	
	Description	
	PrimaConnex Implant Analogs	
45140K	SD, 1-Pack	SD, 1-Pack
45143K	SD, 10-Pack	SD, 10-Pack
45141K	RD, 1-Pack	RD, 1-Pack
45142K	WD, 1-Pack	WD, 1-Pack
	Screws 螺絲	
	Abutment Screw	連接物螺絲
45060K	SD/RD/WD, Replacement Abutment Screw	SD/RD/WD
	Abutment Lab Screw	連接物螺絲
45061K	SD/RD/WD, Abutment Lab Screw (10-Pack)	SD/RD/WD
	Titanium Esthetic Contour Abutments With Screw	美觀型連接物及螺絲
45030K	SD, Straight, 5.0mm Flare, 1mm Cuff	SD, Straight, 5.0mm Flare, 1mm Cuff
45031K	SD, Straight, 5.0mm Flare, 3mm Cuff	SD, Straight, 5.0mm Flare, 3mm Cuff
45032K	SD, 15° Angled, 5.0mm Flare, 1mm Cuff	SD, 15° Angled, 5.0mm Flare, 1mm Cuff
45033K	SD, 15° Angled, 5.0mm Flare, 3mm Cuff	SD, 15° Angled, 5.0mm Flare, 3mm Cuff
45034K	RD, Straight, 6.0mm Flare, 1mm Cuff	RD, Straight, 6.0mm Flare, 1mm Cuff
45035K	RD, 6.0mm Straight, Flare, 3mm Cuff	RD, 6.0mm Straight, Flare, 3mm Cuff
45036K	RD, 15° Angled, 6.0mm Flare, 1mm Cuff	RD, 15° Angled, 6.0mm Flare, 1mm Cuff
45037K	RD, 15° Angled, 6.0mm Flare, 3mm Cuff	RD, 15° Angled, 6.0mm Flare, 3mm Cuff
45038K	WD, Straight, 6.0mm Flare, 1mm Cuff	WD, Straight, 6.0mm Flare, 1mm Cuff
45039K	WD, Straight, 6.0mm Flare, 3mm Cuff	WD, Straight, 6.0mm Flare, 3mm Cuff
45040K	WD, 15° Angled, 6.0mm Flare, 1mm Cuff	WD, 15° Angled, 6.0mm Flare, 1mm Cuff
	UCLA Abutment System UCLA牙喬系統	
	UCLA Gold/Plastic Sleeves (with screw):	UCLA牙喬系統 金屬/塑膠及螺絲連接物
45100K	SD, Locking	SD, Locking
45101K	SD, Non-Locking	SD, Non-Locking
45102K	RD, Locking	RD, Locking
45103K	RD, Non-Locking	RD, Non-Locking
45104K	WD, Locking	WD, Locking
45105K	WD, Non-Locking	WD, Non-Locking
	Multi-Unit Abutment System 多重選擇牙橋系統	
	Multi-Unit Abutment System Abutments:	
45150K	SD, 1mm Cuff	SD, 1mm Cuff
45151K	SD, 2mm Cuff	SD, 2mm Cuff
45152K	SD, 4mm Cuff	SD, 4mm Cuff
45153K	RD, 1mm Cuff	RD, 1mm Cuff
45154K	RD, 2mm Cuff	RD, 2mm Cuff
45155K	RD, 4mm Cuff	RD, 4mm Cuff
45156K	WD, 1mm Cuff	WD, 1mm Cuff
45157K	WD, 2mm Cuff	WD, 2mm Cuff

	Gold Sleeves for Multi-Unit Abutments	
45161K	SD/RD	SD/RD
45168K	WD	WD
	Waxing Pin for Multi-Unit Abutments	
45163K	SD/RD/WD	SD/RD/WD
	Coping Screw for Multi-Unit Abutments	
45164K	SD/RD/WD	SD/RD/WD
	Zest[®] Locator[®] Abutment System 全口假牙系統	
	Locator Abutment Assemblies	配件
45258K	SD, 2mm Cuff	SD, 2mm Cuff
45259K	SD, 3mm Cuff	SD, 3mm Cuff
45260K	SD, 4mm Cuff	SD, 4mm Cuff
45261K	RD, 2mm Cuff	RD, 2mm Cuff
45262K	RD, 3mm Cuff	RD, 3mm Cuff
45263K	RD, 4mm Cuff	RD, 4mm Cuff
45264K	WD, 2mm Cuff	WD, 2mm Cuff
45265K	WD, 3mm Cuff	WD, 3mm Cuff
	Locator Abutment Accessories	配件
45266K	Replacement Male, White, Standard Retention (4-Pack)	配件
45267K	Replacement Male, Pink, LR (Light Retention), (4-Pack)	配件
45269K	Replacement Male, Green, ER, (Extended Range), (4-Pack)	配件
45270K	Female Analog (4-Pack)	Female Analog (4-Pack)
45271K	Impression Coping (4-Pack)	Impression Coping (4-Pack)
45273K	Core Tool	Core Tool
45274K	Torque Wrench Driver	Torque Wrench Driver
	Snap Abutment System Snap連接系統	
	Snap Abutment Assemblies, with Low Retention Insert and Housing	
45250K	SD, 2mm Cuff	SD, 2mm Cuff
45251K	SD, 3mm Cuff	SD, 3mm Cuff
45252K	SD, 4mm Cuff	SD, 4mm Cuff
45253K	RD, 2mm Cuff	RD, 2mm Cuff
45254K	RD, 3mm Cuff	RD, 3mm Cuff
45255K	RD, 4mm Cuff	RD, 4mm Cuff
45256K	WD, 2mm Cuff	WD, 2mm Cuff
45257K	WD, 3mm Cuff	WD, 3mm Cuff
	Snap Abutment Accessories:	配件
L2804-01K	Snap Abutment Analog	Snap Abutment Analog
40003K	Retention Inserts, Low, Red (6-Pack)	Retention Inserts, Low, Red (6-Pack)
40004K	Retention Inserts, Medium, Yellow (6-Pack)	配件
40005K	Retention Inserts, High, Blue (6-Pack)	Retention Inserts, High, Blue (6-Pack)
40007K	Retention Insert Seating Tool	Retention Insert Seating Tool
L2101-96-16K	Overdenture Abutment Driver Tip for Torque Wrench (8mm shaft)	配件
	Surgical Kits and Drill & Tap Sets 手術器械盒和鑽頭	
	Description	
	Surgical Kit for PrimaSolo and PrimaConnex Implants (see catalog for full listing of included items)	
15249K	Straight Drill and Tap Set	直式鑽頭
	Initial and Tapered Drills for Tapered Implants 最初及錐狀鑽頭用於錐狀式植體	
	Initial Drills	
15200K	1.5mm x 15mm Initial Drill	1.5mm x 15mm Initial Drill
15201K	1.8mm x 11.5mm Initial Drill	1.8mm x 11.5mm Initial Drill
15202K	1.8mm x 15mm Initial Drill	1.8mm x 15mm Initial Drill

	Drills for Tapered Implants, Irrigated	鑽頭用於錐狀式植體
15203K	3.0mm x 13mm	3.0mm x 13mm
15204K	3.0mm x 15mm	3.0mm x 15mm
15205K	3.5mm x 10mm	3.5mm x 10mm
15206K	3.5mm x 11.5mm	3.5mm x 11.5mm
15207K	3.5mm x 13mm	3.5mm x 13mm
15208K	3.5mm x 15mm	3.5mm x 15mm
15209K	4.1mm x 10mm	4.1mm x 10mm
15210K	4.1mm x 11.5mm	4.1mm x 11.5mm
15211K	4.1mm x 13mm	4.1mm x 13mm
15212K	4.1mm x 15mm	4.1mm x 15mm
15213K	5.0mm x 10mm	5.0mm x 10mm
15214K	5.0mm x 11.5mm	5.0mm x 11.5mm
15215K	5.0mm x 13mm	5.0mm x 13mm
	Removable Drill Stops for Tapered Drills	可拆卸鑽頭用於錐狀式植體
15220K	3.0mm	3.0mm
15221K	3.5mm	3.5mm
15222K	4.1mm	4.1mm
15223K	5.0mm	5.0mm
	Initial and Straight Drills for Straight Implants 最初及直狀鑽頭用於直狀式植體	
	Drills for Straight Implants, Irrigated	鑽頭用於直狀式植體
15250K	2.8mm x 11.5mm	2.8mm x 11.5mm
15251K	2.8mm x 15mm	2.8mm x 15mm
15252K	3.3mm x 11.5mm	3.3mm x 11.5mm
15253K	3.3mm x 15mm	3.3mm x 15mm
15254K	4.2mm x 11.5mm	4.2mm x 11.5mm
15255K	4.2mm x 13mm	4.2mm x 13mm
	Step Drills for Straight Implants, Irrigated	鑽頭用於直狀式植體
15256K	2.8mm/3.3mm	2.8mm/3.3mm
15257K	3.3mm/4.2mm	3.3mm/4.2mm
	Countersink Drills for Straight Implants, Irrigated	鑽頭用於直狀式植體
15258K	SD	SD
15259K	RD	RD
15260K	WD	WD
	Surgical Taps for Tapered Implants 供牙用於錐狀植體	
	Taps for Tapered Implants, Irrigated	鑽頭
15216K	3.0mm	3.0mm
15217K	3.5mm x 10/11.5mm	3.5mm x 10/11.5mm
15247K	3.5mm x 13/15mm	3.5mm x 13/15mm
15218K	4.1mm x 10/11.5mm	4.1mm x 10/11.5mm
15248K	4.1mm x 13/15mm	4.1mm x 13/15mm
	Surgical Taps for Straight Implants	
	Taps for Straight Implants, Irrigated	鑽頭
15261K	3.3mm x 11.5mm	3.3mm x 11.5mm
15262K	3.3mm x 15mm	3.3mm x 15mm
15263K	4.0mm x 11.5mm	4.0mm x 11.5mm
15264K	4.0mm x 15mm	4.0mm x 15mm
15265K	5.0mm x 13mm	5.0mm x 13mm
	Surgical and Restorative Instrumentation 廣復	
	Abutment Try-In Set for PrimaSolo One-Piece Implants	廣復
15235K	PrimaSolo Abutment Try-In Set	PrimaSolo Abutment Try-In Set
	Parallel Pins	廣復
15226K	Parallel Pin, 1.5mm/1.8mm (2-Pack)	Parallel Pin, 1.5mm/1.8mm (2-Pack)
45045K	Parallel Pin, 1.5mm/1.8mm, 15° (2-Pack)	Parallel Pin, 1.5mm/1.8mm, 15° (2-Pack)
	Implant Drivers for PrimaSolo One-Piece Implants	適用於一件式植體
15227K	3.0	3.0
15228K	SD	SD
15229K	RD	RD
15230K	WD	WD
	Quad Drivers	
15272K	Swivel Head, Short	Swivel Head, Short
15273K	Swivel Head, Long	Swivel Head, Long
15274K	Latch-Type, Short	Latch-Type, Short
15275K	Latch-Type, Long	Latch-Type, Long
15276K	Driver Tip for Torque Wrench, Short (8mm shaft)	配件
15277K	Driver Tip for Torque Wrench, Long (15mm shaft)	配件
	Surgical and Restorative Instrumentation 廣復	
15224K	Ratchet Adapter	配件
15271K	Surgical Hand Driver	配件
15225K	Depth Probe	配件





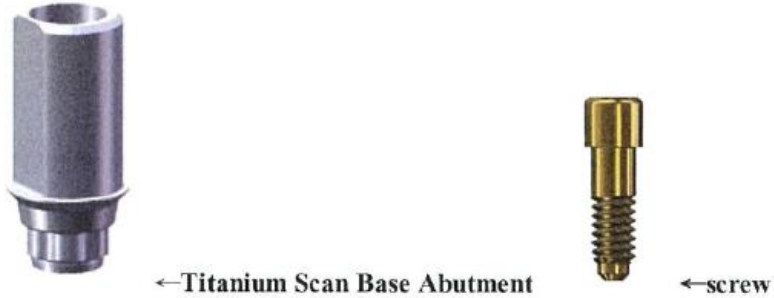











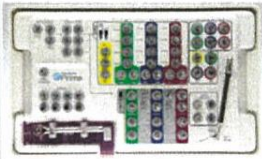

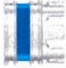




	Description	中譯
Cat.No.		
Stage-1	Stage-1	
42017K	STAGE-1 Snap Abutment,RDS,0mm Cuff	Snap連接系統配件
42018K	STAGE-1 Snap Abutment,RDS,2mm Cuff	Snap連接系統配件
42019K	STAGE-1 Snap Abutment,WDS,0mm Cuff	Snap連接系統配件
42020K	STAGE-1 Snap Abutment,WDS,2mm Cuff	Snap連接系統配件
40003K	Retention Inserts,Low Retention,Red(6-pack)	Snap連接系統配件
40004K	Retention Inserts,Medium Retention,Yellow(6-pack)	Snap連接系統配件
40005K	Retention Inserts,High Retention,Blue(6-pack)	Snap連接系統配件
40007K	Retention Inserts Seating Tool	Snap連接系統配件
42021K	RDS COC Temp/Healing Cap 4.0mm	臨時/癒合帽
42022K	RDS COC Temp/Healing Cap 5.5mm	臨時/癒合帽
42023K	RDS COC Temp/Healing Cap 7.0mm	臨時/癒合帽
42024K	WDS COC Temp/Healing Cap 4.0mm	臨時/癒合帽
42025K	WDS COC Temp/Healing Cap 5.5mm	臨時/癒合帽
42026K	WDS COC Temp/Healing Cap 7.0mm	臨時/癒合帽
Cat.No.	Restore	
41000K	RESTORE Snap Abutment,SD,2mm Cuff	Snap連接系統配件
41001K	RESTORE Snap Abutment,SD,3mm Cuff	Snap連接系統配件
41002K	RESTORE Snap Abutment,SD,4mm Cuff	Snap連接系統配件
41003K	RESTORE Snap Abutment,RD,2mm Cuff	Snap連接系統配件
41004K	RESTORE Snap Abutment,RD,3mm Cuff	Snap連接系統配件
41005K	RESTORE Snap Abutment,RD,4mm Cuff	Snap連接系統配件
41006K	RESTORE Snap Abutment,WD,2mm Cuff	Snap連接系統配件
41007K	RESTORE Snap Abutment,WD,3mm Cuff	Snap連接系統配件
Cat.No.	Prima	
15282K	PrimaConnex Implant Driver,SD,Ratchet	適用於兩件式植體/螺絲鉗
15283K	PrimaConnex Implant Driver,RD,Ratchet	適用於兩件式植體/螺絲鉗
15284K	PrimaConnex Implant Driver,WD,Ratchet	適用於兩件式植體/螺絲鉗
45172K	PrimaSolo 3.0 Implant Analog	連接體替代物
45173K	PrimaSolo SD Implant Analog	連接體替代物
45174K	PrimaSolo RD Implant Analog	連接體替代物
45175K	PrimaSolo WD Implant Analog	連接體替代物

PrimaSolo RBM		
	Keystone's CFG	
Cat.No.		中譯
	PrimaSolo and PrimaConnex Implants, RBM Surface Modification	
15400K	PrimaSolo One-Piece Tapered Implant, 3.0×13, Var RBM	Solo錐型植體3.0×13
15401K	PrimaSolo One-Piece Tapered Implant, 3.0×15, Var RBM	Solo錐型植體3.0×15
15402K	PrimaSolo One-Piece Tapered Implant, 3.5×10, Var RBM	Solo錐型植體3.5×10
15403K	PrimaSolo One-Piece Tapered Implant, 3.5×11.5, Var RBM	Solo錐型植體3.5×11.5
15404K	PrimaSolo One-Piece Tapered Implant, 3.5×13, Var RBM	Solo錐型植體3.5×13
15405K	PrimaSolo One-Piece Tapered Implant, 3.5×15, Var RBM	Solo錐型植體3.5×15
15410K	PrimaSolo One-Piece Tapered Implant, 5.0×10, Var RBM	Solo錐型植體5.0×10
15411K	PrimaSolo One-Piece Tapered Implant, 5.0×11.5, Var RBM	Solo錐型植體5.0×11.5
15412K	PrimaSolo One-Piece Tapered Implant, 5.0×13, Var RBM	Solo錐型植體5.0×13
15413K	SD 3.5mm × 10mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體3.5×10
15414K	SD 3.5mm × 11.5mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體3.5×11.5
15415K	SD 3.5mm × 13mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體3.5×13
15416K	SD 3.5mm × 15mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體3.5×15
15417K	RD 4.1mm × 10mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體4.1×10
15418K	RD 4.1mm × 11.5mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體4.1×11.5
15419K	RD 4.1mm × 13mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體4.1×13
15421K	WD 5.0mm × 10mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體5.0×10
15422K	WD 5.0mm × 11.5mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體5.0×11.5
15423K	WD 5.0mm × 13mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體5.0×13
15424K	WD 5.0mm × 15mm PrimaConnex Tapered Implant, Var RBM	Connex錐型植體5.0×15
15425K	SD 3.3mm × 10mm PrimaConnex Straight Implant, Var RBM	Connex直型植體3.3×10
15426K	SD 3.3mm × 11.5mm PrimaConnex Straight Implant, Var RBM	Connex直型植體3.3×11.5
15427K	SD 3.3mm × 13mm PrimaConnex Straight Implant, Var RBM	Connex直型植體3.3×13
15428K	SD 3.3mm × 15mm PrimaConnex Straight Implant, Var RBM	Connex直型植體3.3×15
15429K	RD 4.0mm × 8mm PrimaConnex Straight Implant, Var RBM	Connex直型植體4.0×8
15430K	RD 4.0mm × 10mm PrimaConnex Straight Implant, Var RBM	Connex直型植體4.0×10
15431K	RD 4.0mm × 11.5mm PrimaConnex Straight Implant, Var RBM	Connex直型植體4.0×11.5
15432K	RD 4.0mm × 13mm PrimaConnex Straight Implant, Var RBM	Connex直型植體4.0×13
15433K	RD 4.0mm × 15mm PrimaConnex Straight Implant, Var RBM	Connex直型植體4.0×15
15434K	WD 5.0mm × 8mm PrimaConnex Straight Implant, Var RBM	Connex直型植體5.0×8
15435K	WD 5.0mm × 10mm PrimaConnex Straight Implant, Var RBM	Connex直型植體5.0×10
15436K	WD 5.0mm × 11.5mm PrimaConnex Straight Implant, Var RBM	Connex直型植體5.0×11.5
15437K	WD 5.0mm × 13mm PrimaConnex Straight Implant, Var RBM	Connex直型植體5.0×13
15406K	PrimaSolo One-Piece Tapered Implant, 4.1×10 , Var RBM	Solo錐型植體4.1×10
15407K	PrimaSolo One-Piece Tapered Implant, 4.1×11.5 , Var RBM	Solo錐型植體4.1×11.5
15408K	PrimaSolo One-Piece Tapered Implant, 4.1×13 , Var RBM	Solo錐型植體4.1×13
15409K	PrimaSolo One-Piece Tapered Implant, 4.1×15 , Var RBM	Solo錐型植體4.1×15

Titanium Scan Base Abutment (with screw-45060K)		材質	滅菌方式
 <p>←Titanium Scan Base Abutment ←screw</p>			
45901K	Scan Base, Screw-Retained 3.5/3.8 TiLobe Connection	Ti-6Al-4V Grade 5	Gamma
45902K	Scan Base, Screw-Retained 4.1/4.5 TiLobe Connection		
45903K	Scan Base, Screw-Retained 5.0/5.5/6.5 TiLobe Connection		

The pictures of Implant and Abutment made by Keystone Dental, Inc.

15306K - Cover Screws	45141K - Implant Analogs	45060K - Abutment Screw	45061K - Abutment Lab Screw	45034K - Titanium Esthetic Contour Abutments
				
45153K - Multi-Unit Abutment	45158K - Multi-Unit Abutment Healing Cap	45159K - Multi-Unit Abutment Impression Post	45161K - Multi-Unit Abutment Gold Sleeve	45163K - Multi-Unit Abutment Waxing Pin
				
45164K - Multi-Unit Abutment Coping Screw	45260K - Locator Abutment Assembly	45266K - Replacement Male	45270K - Female Analog	45271K - Impression Coping
				
45273K - Core Tool			45274K - Torque Wrench Driver	45253K - Snap Abutment
				

L2804-01K - Snap Abutment Analog	40003K - Retention Inserts	40007K - Retention Insert Seating Tool	L2101-96-16K - Overdenture Abutment Driver Tip for Torque Wrench	15249K - Straight Drill and Tap Set
				
15200K - Initial Drills	15203K - Drills for Tapered Implants	15220K - Removable Drill Stops for Tapered Drills	15250K - Drills for Straight Implants	15256K - Step Drills for Straight Implants
				
15258K - Countersink Drills for Straight Implants	15217K - Taps for Tapered Implants	15261K - Taps for Straight Implants	15226K - Parallel Pin, 1.5mm/1.8mm	45045K - Parallel Pin, 1.5mm/1.8mm, 15°
				
15235K - PrimaSolo Abutment Try-In Set	15227K - Implant Drivers for PrimaSolo One-Piece Implants	15272K - Quad Drivers Swivel Head	15274K - Quad Drivers Latch-Type	15276K - Quad Driver Tip for Torque Wrench
				
15224K - Ratchet Adapter	15271K - Surgical Hand Driver	15225K - Depth Probe		
				

RSR3308-2K - STAGE-1 RBM Implants	S1800-30K - STAGE-1 Cover Screws	S2507-40K - COC Quick Pack	S2503K - COC Abutment Try-in Set	S2432-40-1K - COC Abutment
				
42001K - COC Angled Abutment	42000K - Angled Abutment Screw	S2435K - COC Custom Prep Abutment	S2505K - Implant Analog	S2431-40K - COC Quick Cap
				
S2439K - COC Abutment Temporary Cap	42021K - COC Abutment Temporary/Healing Cap	S2437-00-1K - COC Abutment Plastic Sleeve	S2440-01K - COC Reamer Tool	S2400K - Indexed Abutment
				
S2404K - Abutment Analog	S2406-00K - Indexed Abutment Plastic Sleeve	S2405-00K - Indexed Abutment Gold Sleeve	S2402K - Indexed Abutment Impression Post	S2407-00K - Indexed Abutment Temporary Sleeve
				
S2408K - Indexed Abutment Screw	42017K - Snap Abutment Assembly	S2417-0K - O-Ring Abutment	L2805-01K - O-Ring Accessories Abutment Analog	L2810-01K - O-Ring Accessories Keeper, Titanium
				

S1101-28-10K - Irrigated Drills	S1100-28-10K - Non-Irrigated Drills	S1500-33-10K - Irrigated Taps	L1702-01K - Surgical Ratchet	S1701-01K - Ratchet Adapter
				
S1601-01K - Placement Head	S1700K - Stabilizing Wrench	S1400-02K - Headpiece Adapter	S1402-01K - Surgical Hand Driver	L1404-01K - Locking Drill Extender
				
S1802K - Surgical Depth Probe	S1803-28K - Depth Gauge	R9121K - .048" Hex Driver, Swivel Head	L1400-48-23K - .048" Hex Driver, Latch-Type	S2420-1K - COC Abutment Driver
				
L2100-62-19K - .062" Hex Driver, Swivel Head	L1404-96-20K - Overdenture Abutment Driver, Latch-Type	L1809-34-01K - Straight Cover Screw	L1810-40-01K - Headless Cover Screw	R9633-34-02K - Healing Abutment
				
R9593-34-01K - Non-Hexed Implant Impression Posts	R9893-34K - Implant Analog	R9290-34-02K - Straight Locking COC Abutment	R2302-42-02K - Straight Non-Locking COC Abutment	R9203-34-48K - COC Abutment Screw
				

L2300-33-01K - Conical Abutment	R9212-40K - Conical Abutment Healing Cap	R9530-40-01K - Conical Abutment Impression Post	R9845-40K - Conical Abutment Analog	R9211-40-01K - Conical Abutment Plastic Sleeve
				
R9200-40-48K - Conical Abutment Fastening Screw	R9086-40-03K - Standard Abutment	R9284-40K - Standard Abutment Healing Cap	R9886-40K - Standard Abutment Analog	41000K - Snap Abutment Assembly
				
L2304-42-02K - O-Ring Abutment	L2805-01K - Abutment Analog	L2810-01K - Keeper, Titanium	L1405-23-00K - Round Bur	L1101-28-10K - Surgical Drill for 3.3mm Implants
				
L1300-20-30K - Pilot Drill	L1502-33-13K - 3.3mm Surgical Tap, Irrigated	L1411-01K - Bone Profiler System, Guide Pin	R9192-33-05K - Narrow Profile Placement Head	L1813-01K - Placement Head Assembly Block
				
L1709-01K - Stabilizing Wrench	L1702-01K - Surgical Ratchet	L1806-04K - Ratchet Adapter, 8.0mm	L1806-05K - Ratchet/Tap Adapter, 12mm	L2100-85-19K - Square Driver, Swivel Head
				

L1400-85-23K - Square Driver, Latch-Type	ISS1118-1K - Driver Top for Accu-Torque	L1812-01K - Component Pack	L2802-01K - Reamer Handle	L2800-01K - Reamer
				
ISS185BK - Hader Castable Bar	ISS185HPK - Hader Processing Clip	ISS185HRK - Hader Final Clip	ISS526HK - Hader Metal Housing Clip	L1406-02K - Hand Adapter for NP Placement Heads
				
L1406-01K - Handpiece Adapter	L1414-01K - Surgical Hand Driver	L1404-01K - Locking Drill Extender	L1800-22-02K - Irrigation Port Cleaner	I3280-40K - Tissue Measuring Gauge
				
ISS140K - Disposable Rissue Punch	L1805-20-00K - Straight Parallel Pin	R2402-01K - Gold Bar Cylinder	L1700-72K - Internal Thread Tap for Implants	45172K - Lifecore PrimaSolo Implant Analog
				
15282K - Lifecore PrimaConnex Ratchet Implant Driver				
				

45901K

Scan Base, Screw-Retained
3 5/32 TiLobe® Connection

Scan Base, Screw-Retained
3 5/32 TiLobe® Connection

REF 45901K LOT 18321-1
Keystone Dental, Inc. 1-781-328-3490

STERILE R
2019-01
LOT 18321-1

CE 0086

Keystone DENTAL
Keystone Dental, Inc.
144 Middlesex Turnpike
Burlington, MA 01803 USA
1-781-328-3490
4409-0228 Rev 9/2012

Only Made in USA
EC REP
Keystone Dental S.p.A.
Via A. Fleming, 19
37135 Verona, Italy
+39-045-8230294



45902K

Scan Base, Screw-Retained
4 1/4 5 TiLobe® Connection

Scan Base, Screw-Retained
4 1/4 5 TiLobe® Connection


REF 45902K LOT 18322-1
Keystone Dental, Inc. 1-781-328-3490

STERILE R
2019-01
LOT 18322-1

CE 0086

Keystone DENTAL
Keystone Dental, Inc.
144 Middlesex Turnpike
Burlington, MA 01803 USA
1-781-328-3490
4409-0228 Rev 9/2012

Only Made in USA
EC REP
Keystone Dental S.p.A.
Via A. Fleming, 19
37135 Verona, Italy
+39-045-8230294

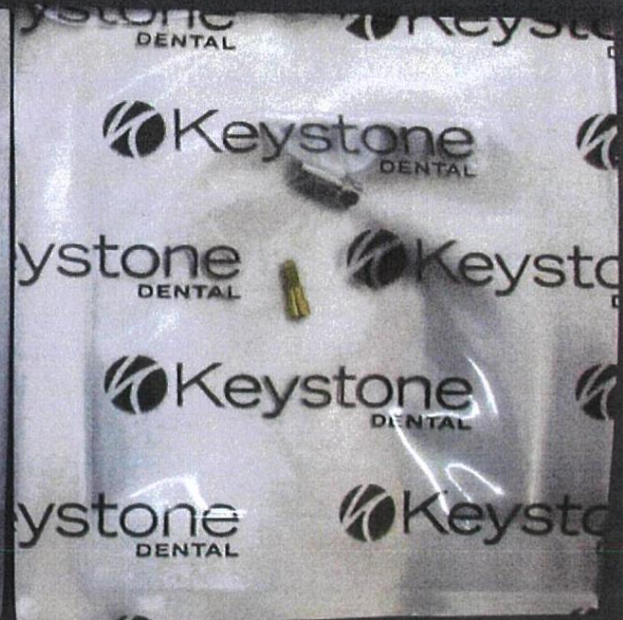

45903K

