

“凱石”牙科傑耐斯牙橋

“Keystone” Dental Genesis Abutment

衛部醫器輸字第 026627 號

使用前請務必詳閱原廠之使用說明書並遵照指示使用。

產品敘述：

本產品包括支台體及相關配件，須搭配“凱石”牙科傑耐斯植體使用。

本產品材質部分支台體為鈦合金製成，並經 AnaTite 表面處理方式，另一部份支台體為鈦合金和塑膠或金合金和塑膠製成。支台體需搭配鈦合金材質之螺絲，將贗復產品固定於植體。本產品包含多種尺寸，可搭配相對應之植體平臺。

產品用途：

本產品搭配植體使用時，適用於一階或二階手術過程並可植入於部份及全口無牙的上下顎骨。本產品適用於單顆及多顆贗復產品，以重建病患的咀嚼功能及美觀。本產品搭配植體使用時，可植入於缺牙區或拔牙後立即植入。當產品與骨頭建立良好的初期穩定性，可適當咬合且立即受力。

禁忌症：

- 甲狀腺機能亢進、糖尿病、惡性腫瘤、腎臟病、肝臟疾病、高血壓、白血病、嚴重的心血管疾病、肝炎、免疫抑制的失調、膠原蛋白和骨頭疾病、或是其他嚴重疾病。
- 病患對鈦合金過敏
- 病患的齒槽骨未有足夠空間植入植體
- 全身性感染、局部口腔及呼吸道感染。

警告：

- 植牙手術並非沒有風險。醫師有義務告知病患植牙手術的風險。
- 病患的術前評估是必要的，其可決定造成病患風險的因素，或是影響骨頭及軟組織癒合的過程。
- 注意配件不可被病患吞嚥。
- 下列狀況本產品可能無法有效發揮功能：慢性出血問題、精神損傷、代謝性骨病或結締組織疾病、類皮質激素的治療、心血管疾病、吸菸者、無法控制的糖尿病、化學藥劑治療、慢性腎臟病、不良的口腔衛生、磨牙或酗酒。
- 在正常使用下，植體也可能發生斷裂或失敗。
- 醫師應慎選擇搭配之植體尺寸規格，可降低植牙失敗的風險。

- 本產品含金屬材質，當使用含電手術器械或雷射，可能造成金屬材質電或熱傳導。
- 植體移動、骨質流失或慢性感染可能導致植體植入失敗。
- 如果產品表面有損壞，請勿使用。
- 本產品僅供單次病患使用，不可重新滅菌或重複使用。
- 不可任意更改產品。

預防：

- 只能受過專業訓練的合格醫師方能使用本產品，不適當植牙技術會造成植體植入失敗或骨質流失。
- 植體植入前應適當執行臨床及 X 光影像評估。
- 植入前，應決定植體置入解剖位置與確認具足夠的骨質。
- 本產品使用前請務必詳閱原廠之使用說明書。

副作用：

可能造成的併發症：

- 感染發炎
- 病人的不適(疼痛)
- 骨整合不足，植體的移動
- 植體斷裂
- 上顎脈竇穿孔
- 舌穿孔
- 支台體的螺絲鬆脫
- 骨頭流失
- 局部軟組織惡化

未滅菌產品之清洗滅菌相關程序

建議滅菌方法及條件

高溫高壓滅菌鍋:

蒸氣滅菌重力循環:134°C (~273°F) 暴露 20 分鐘/乾燥 40 分鐘

蒸氣滅菌前真空循環: 134°C (~273°F) 暴露 4 分鐘/乾燥 40 分鐘

警告：每一個循環請勿超過:140°C (~284°F)

警告：使用高溫高壓滅菌鍋滅菌，滅菌前，需把產品分別置於手術托盤、密封滅菌袋或手術巾內。

本產品清潔及滅菌程序

部份產品提供無菌包裝，經 Gamma 滅菌處理。請依照個別包裝標籤去判別本產品是否經滅菌處理。在使用前，未滅菌產品需經清潔及滅菌程序。清潔程序建議使用清潔劑與水之混合溶液搭配超音波清洗儀器。關於滅菌程序，請參照以下的說明：

系列	滅菌方式
UCLA Gold/Plastic Abutment* Ti Temporary Abutment	未滅菌產品之建議滅菌方式及條件： 蒸氣滅菌，重力循環：134°C (~273°F) 暴露 20 分鐘/乾燥 40 分鐘 蒸氣滅菌前真空循環：134°C (~273°F) 暴 露 4 分鐘/乾燥 40 分鐘
Healing Abutment Temporary Abutment Final Abutment Screw Snap Abutment Locator Abutment Esthetic Contour Ti Abutment Quick Abutment Quick Abutment Temporary/Healing cap	無菌包裝，已經 Gamma 滅菌處理。

*警告：塑膠材質產品，請勿用高溫高壓滅菌處理。建議將從技工所完成的 UCLA 支台體產品，請參照上表內 UCLA Gold/Plastic Abutment 程序滅菌。

產品型號:見附錄一。

製造廠名稱: Keystone Dental, Inc.

製造廠地址: 144 Middlesex Turnpike Burlington, MA 01803, U.S.A.




藥商名稱: 杏昌生技股份有限公司

藥商地址: 新北市三重區重新路五段 609 巷 2 號 9 樓之 2

附錄一：產品型號

Genesis Healing Abutments		材質	滅菌方式		
 Healing Abutment(經 AnaTite 表面處理)					
G21103	Healing Abutment, 4.0mm Contour, 3mm Cuff For 3.8mm Genesis Implants	鈦合金	Gamma		
G21104	Healing Abutment, 4.0mm Contour, 5mm Cuff For 3.8mm Genesis Implants				
G21105	Healing Abutment, 5.0mm Contour, 3mm Cuff For 3.8mm Genesis Implants				
G21106	Healing Abutment, 5.0mm Contour, 5mm Cuff For 3.8mm Genesis Implants				
G21107	Healing Abutment, 5.0mm Contour, 3mm Cuff For 4.5mm Genesis Implants				
G21108	Healing Abutment, 5.0mm Contour, 5mm Cuff For 4.5mm Genesis Implants				
G21109	Healing Abutment, 6.0mm Contour, 3mm Cuff For 4.5mm Genesis Implants				
G21110	Healing Abutment, 6.0mm Contour, 5mm Cuff For 4.5mm Genesis Implants				
G21111	Healing Abutment, 5.0mm Contour, 3mm Cuff For 5.5mm Genesis Implants				
G21112	Healing Abutment, 5.0mm Contour, 5mm Cuff For 5.5mm Genesis Implants				
G21113	Healing Abutment, 6.0mm Contour, 3mm Cuff For 5.5/6.5mm Genesis Implants				
G21114	Healing Abutment, 6.0mm Contour, 5mm Cuff For 5.5/6.5mm Genesis Implants				
Genesis Temporary Abutments (with screw-G71125, Occlusal plug)				材質	滅菌方式
 Temporary Abutment screw					
G71130	Temporary Abutment, 4.0mm Contour, Locking For 3.8mm Genesis Implants	Top-PMMA Down-鈦合金	Gamma		
G71131	Temporary Abutment, 4.0mm Contour, Non-Locking For 3.8mm Genesis Implants				
G71132	Temporary Abutment, 5.0mm Contour, Locking For 3.8mm Genesis Implants				
G71133	Temporary Abutment, 5.0mm Contour, Non-Locking For 3.8mm Genesis Implants				
G71134	Temporary Abutment, 5.0mm Contour, Locking For 4.5mm Genesis Implants				
G71135	Temporary Abutment, 5.0mm Contour, Non-Locking For 4.5mm Genesis Implants				
G71136	Temporary Abutment, 6.0mm Contour, Locking For 4.5mm Genesis Implants				
G71137	Temporary Abutment, 6.0mm Contour, Non-Locking For 4.5mm Genesis Implants				
G71138	Temporary Abutment, 5.0mm Contour, Locking For 5.5mm Genesis Implants				
G71139	Temporary Abutment, 5.0mm Contour, Non-Locking For 5.5mm Genesis Implants				
G71140	Temporary Abutment, 6.0mm Contour, Locking For 5.5/6.5mm Genesis Implants				
G71141	Temporary Abutment, 6.0mm Contour, Non-Locking For 5.5/6.5mm Genesis Implants				
Genesis Ti Temporary Abutments(with screw-G71125, Occlusal plug)				材質	滅菌方式

			
Ti Temporary Abutment(經 AnaTite 表面處理) screw			
G71144	Ti Temporary Abutment, Locking For 3.8mm Genesis Implants	鈦合金	Non-Sterile
G71145	Ti Temporary Abutment, Non-Locking For 3.8mm Genesis Implants		
G71146	Ti Temporary Abutment, Locking For 4.5mm Genesis Implants		
G71147	Ti Temporary Abutment, Non-Locking For 4.5mm Genesis Implants		
G71148	Ti Temporary Abutment, Locking For 5.5/6.5mm Genesis Implants		
G71149	Ti Temporary Abutment, Non-Locking For 5.5/6.5mm Genesis Implants		
Genesis Esthetic Contour Ti Abutments (with screw-G71125)			
		材質	滅菌方式
Esthetic Contour Ti Abutments(經 AnaTite 表面處理) screw			
G71200	Esthetic Contour Ti, Str, 4.0mm Contour, 1mm Cuff For 3.8mm Genesis Implants	鈦合金	Gamma
G71201	Esthetic Contour Ti, Str, 4.0mm Contour, 2mm Cuff For 3.8mm Genesis Implants		
G71202	15°Angled Esthetic Contour Ti, 4.0mm Contour, 1mm Cuff For 3.8mm Genesis Implants		
G71203	15°Angled Esthetic Contour Ti, 4.0mm Contour, 2mm Cuff For 3.8mm Genesis Implants		
G71204	Esthetic Contour Ti, Str, 5.0mm Contour, 1mm Cuff For 3.8mm Genesis Implants		
G71205	Esthetic Contour Ti, Str, 5.0mm Contour, 3mm Cuff For 3.8mm Genesis Implants		
G71206	15°Angled Esthetic Contour Ti, 5.0mm Contour, 1mm Cuff For 3.8mm Genesis Implants		
G71207	15°Angled Esthetic Contour Ti, 5.0mm Contour, 3mm Cuff For 3.8mm Genesis Implants		
G71208	Esthetic Contour Ti, Str, 5.0mm Contour, 0.5mm Cuff For 4.5mm Genesis Implants		
G71209	Esthetic Contour Ti, Str, 5.0mm Contour, 1.5mm Cuff For 4.5mm Genesis Implants		
G71210	15°Angled Esthetic Contour Ti, 5.0mm Contour, 0.5mm Cuff For 4.5mm Genesis Implants		
G71211	15°Angled Esthetic Contour Ti, 5.0mm Contour, 1.5mm Cuff For 4.5mm Genesis Implants		
G71212	Esthetic Contour Ti, Str, 6.0mm Contour, 1mm Cuff For 4.5mm Genesis Implants		
G71213	Esthetic Contour Ti, Str, 6.0mm Contour, 3mm Cuff For 4.5mm Genesis Implants		
G71214	15oAngled Esthetic Contour Ti, 6.0mm Contour, 1mm Cuff For 4.5mm Genesis Implants		

G71215	15°Angled Esthetic Contour Ti, 6.0mm Contour, 3mm Cuff For 4.5mm Genesis Implants	鈦合金	Gamma
G71216	Esthetic Contour Ti, Str, 5.0mm Contour, 0.5mm Cuff For 5.5mm Genesis Implants		
G71217	Esthetic Contour Ti, Str, 5.0mm Contour, 1.5mm Cuff For 5.5mm Genesis Implants		
G71219	15°Angled Esthetic Contour Ti, 5.0mm Contour, 1.5mm Cuff For 5.5mm Genesis Implants		
G71220	Esthetic Contour Ti, Str, 6.0mm Contour, 1mm Cuff For 5.5/6.5mm Genesis Implants		
G71221	Esthetic Contour Ti, Str, 6.0mm Contour, 3mm Cuff For 5.5/6.5mm Genesis Implants		
G71222	15°Angled Esthetic Contour Ti, 6.0mm Contour, 1mm Cuff For 5.5/6.5mm Genesis Implants		
Final Abutment Screws			
 <p>Screw</p>		材質	滅菌方式
G71125	Replacement Final Abutment Screw, sterile(M1.8)	鈦合金	Gamma
Genesis UCLA Gold/Plastic Sleeve (with screw-G71125, Occlusal plug)			
 <p>Genesis UCLA Gold/Plastic Sleeve</p>  <p>screw</p>		材質	滅菌方式
G71231	UCLA Gold/Plastic Sleeve, Locking, 1mm Cuff For 3.8mm Genesis Implants	Down-gold alloy/ Top-Acetal	Non-Sterile
G71232	UCLA Gold/Plastic Sleeve, Non-Locking, 1mm Cuff For 3.8mm Genesis Implants		
G71233	UCLA Gold/Plastic Sleeve, Locking, 1mm Cuff For 4.5mm Genesis Implants		
G71234	UCLA Gold/Plastic Sleeve, Non-Locking, 1mm Cuff For 4.5mm Genesis Implants		
G71235	UCLA Gold/Plastic Sleeve, Locking, 1mm Cuff For 5.5/6.5mm Genesis Implants		
G71236	UCLA Gold/Plastic Sleeve, Non-Locking, 1mm Cuff For 5.5/6.5mm Genesis Implants		
Genesis Quick Abutment System			
Genesis Quick Packs(includes Abutment with screw, Impression cap, Abutment analog, Temporary/Healing cap and Plastic Sleeve-內容物包含支台樁、印模蓋、支台樁仿植體、暫時/癒合蓋及塑膠套筒)		材質	滅菌方式



Quick abutment(經 AnaTite 表面處理)



screw



Temporary/Healing Cap



Quick cap



analog






plastic sleeve






<p>G71400</p>	<p>Quick Packs, 5.0mm Contour, 1mm Cuff For 3.8mm Genesis Implants 內容物： G71434 Quick Abutment (含 screw-G71125) G71462 Temporary/Healing cap G71478 plastic sleeve, ϕ 3.8mm, Locking, 5.0mm Contour (綠) G71468 Abutment analog, ϕ 3.8mm, 5.0mm Contour (綠) G71428 Quick cap, ϕ 3.8mm, 5.0mm Contour (綠)</p>		
<p>G71401</p>	<p>Quick Packs, 5.0mm Contour, 2mm Cuff For 3.8mm Genesis Implants 內容物： G71435 Quick Abutment (含 screw-G71125) G71462 Temporary/Healing cap G71478 plastic sleeve, ϕ 3.8mm, Locking, 5.0mm Contour (綠) G71468 Abutment analog, ϕ 3.8mm, 5.0mm Contour (綠) G71428 Quick cap, ϕ 3.8mm, 5.0mm Contour (綠)</p>	<p>1.Abutment:鈦合金—Gamma 2.Screw:鈦合金—Gamma 3.Temporary Healing Cap:PMMA—Gamma 4.Impression Cap: Polypropylene—Non-Sterile 5.Analog:鈦合金—Non-Sterile 6.Plastic Sleeve: Acetal—Non-Sterile</p>	
<p>G71402</p>	<p>Quick Packs, 5.0mm Contour, 3mm Cuff For 3.8mm Genesis Implants 內容物： G71436 Quick Abutment (含 screw-G71125) G71462 Temporary/Healing cap G71478 plastic sleeve, ϕ 3.8mm, Locking, 5.0mm Contour (綠) G71468 Abutment analog, ϕ 3.8mm, 5.0mm Contour (綠) G71428 Quick cap, ϕ 3.8mm, 5.0mm Contour (綠)</p>		<p>備註：產品各別包裝後，置於紙盒內。</p>
<p>G71403</p>	<p>Quick Packs, 5.0mm Contour, 4mm Cuff For 3.8mm Genesis Implants 內容物： G71437 Quick Abutment (含 screw-G71125) G71462 Temporary/Healing cap</p>		

	<p>G71478 plastic sleeve, ϕ 3.8mm, Locking, 5.0mm Contour (綠)</p> <p>G71468 Abutment analog, ϕ 3.8mm, 5.0mm Contour (綠)</p> <p>G71428 Quick cap, ϕ 3.8mm, 5.0mm Contour (綠)</p>
G71404	<p>Quick Packs, 5.0mm Contour, 0.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71438 Quick Abutment (含 screw-G71125)</p> <p>G71463 Temporary/Healing cap</p> <p>G71480 plastic sleeve, ϕ 4.5mm, Locking, 5.0mm Contour (藍)</p> <p>G71469 Abutment analog, ϕ 4.5mm, 5.0mm Contour (藍)</p> <p>G71429 Quick cap, ϕ 4.5mm, 5.0mm Contour (藍)</p>
G71405	<p>Quick Packs, 5.0mm Contour, 1.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71439 Quick Abutment (含 screw-G71125)</p> <p>G71463 Temporary/Healing cap</p> <p>G71480 plastic sleeve, ϕ 4.5mm, Locking, 5.0mm Contour (藍)</p> <p>G71469 Abutment analog, ϕ 4.5mm, 5.0mm Contour (藍)</p> <p>G71429 Quick cap, ϕ 4.5mm, 5.0mm Contour (藍)</p>
G71406	<p>Quick Packs, 5.0mm Contour, 2.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71440 Quick Abutment (含 screw-G71125)</p> <p>G71463 Temporary/Healing cap</p> <p>G71480 plastic sleeve, ϕ 4.5mm, Locking, 5.0mm Contour (藍)</p> <p>G71469 Abutment analog, ϕ 4.5mm, 5.0mm Contour (藍)</p> <p>G71429 Quick cap, ϕ 4.5mm, 5.0mm Contour (藍)</p>
G71407	<p>Quick Packs, 5.0mm Contour, 4mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71441 Quick Abutment (含 screw-G71125)</p> <p>G71463 Temporary/Healing cap</p> <p>G71480 plastic sleeve, ϕ 4.5mm, Locking, 5.0mm Contour (藍)</p> <p>G71469 Abutment analog, ϕ 4.5mm, 5.0mm Contour (藍)</p> <p>G71429 Quick cap, ϕ 4.5mm, 5.0mm Contour (藍)</p>
G71408	<p>Quick Packs, 6.0mm Contour, 0.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71442 Quick Abutment (含 screw-G71125)</p> <p>G71464 Temporary/Healing cap</p> <p>G71482 plastic sleeve, ϕ 4.5mm, Locking, 6.0mm Contour (藍)</p> <p>G71470 Abutment analog, ϕ 4.5mm, 6.0mm Contour (藍)</p> <p>G71430 Quick cap, ϕ 4.5mm, 6.0mm Contour (藍)</p>
G71409	<p>Quick Packs, 6.0mm Contour, 1.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p>

	<p>G71443 Quick Abutment (含 screw-G71125)</p> <p>G71464 Temporary/Healing cap</p> <p>G71482 plastic sleeve, ϕ 4.5mm, Locking, 6.0mm Contour (藍)</p> <p>G71470 Abutment analog, ϕ 4.5mm, 6.0mm Contour (藍)</p> <p>G71430 Quick cap, ϕ 4.5mm, 6.0mm Contour (藍)</p>
G71410	<p>Quick Packs, 6.0mm Contour, 2.5mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71444 Quick Abutment (含 screw-G71125)</p> <p>G71464 Temporary/Healing cap</p> <p>G71482 plastic sleeve, ϕ 4.5mm, Locking, 6.0mm Contour (藍)</p> <p>G71470 Abutment analog, ϕ 4.5mm, 6.0mm Contour (藍)</p> <p>G71430 Quick cap, ϕ 4.5mm, 6.0mm Contour (藍)</p>
G71411	<p>Quick Packs, 6.0mm Contour, 4mm Cuff For 4.5mm Genesis Implants</p> <p>內容物：</p> <p>G71445 Quick Abutment (含 screw-G71125)</p> <p>G71464 Temporary/Healing cap</p> <p>G71482 plastic sleeve, ϕ 4.5mm, Locking, 6.0mm Contour (藍)</p> <p>G71470 Abutment analog, ϕ 4.5mm, 6.0mm Contour (藍)</p> <p>G71430 Quick cap, ϕ 4.5mm, 6.0mm Contour (藍)</p>
G71412	<p>Quick Packs, 5.0mm Contour, 0.5mm Cuff For 5.5mm Genesis Implants</p> <p>內容物：</p> <p>G71446 Quick Abutment (含 screw-G71125)</p> <p>G71465 Temporary/Healing cap</p> <p>G71484 plastic sleeve, ϕ 5.5mm, Locking, 5.0mm Contour (紫)</p> <p>G71471 Abutment analog, ϕ 5.5mm, 5.0mm Contour (紫)</p> <p>G71431 Quick cap, ϕ 5.5mm, 5.0mm Contour (紫)</p>
G71413	<p>Quick Packs, 5.0mm Contour, 1.5mm Cuff For 5.5mm Genesis Implants</p> <p>內容物：</p> <p>G71447 Quick Abutment (含 screw-G71125)</p> <p>G71465 Temporary/Healing cap</p> <p>G71484 plastic sleeve, ϕ 5.5mm, Locking, 5.0mm Contour (紫)</p> <p>G71471 Abutment analog, ϕ 5.5mm, 5.0mm Contour (紫)</p> <p>G71431 Quick cap, ϕ 5.5mm, 5.0mm Contour (紫)</p>
G71414	<p>Quick Packs, 5.0mm Contour, 2.5mm Cuff For 5.5mm Genesis Implants</p> <p>內容物：</p> <p>G71448 Quick Abutment (含 screw-G71125)</p> <p>G71465 Temporary/Healing cap</p> <p>G71484 plastic sleeve, ϕ 5.5mm, Locking, 5.0mm Contour (紫)</p> <p>G71471 Abutment analog, ϕ 5.5mm, 5.0mm Contour (紫)</p> <p>G71431 Quick cap, ϕ 5.5mm, 5.0mm Contour (紫)</p>

G71415	<p>Quick Packs, 5.0mm Contour, 4mm Cuff For 5.5mm Genesis Implants 內容物： G71449 Quick Abutment (含 screw-G71125) G71465 Temporary/Healing cap G71484 plastic sleeve, ϕ 5.5mm, Locking, 5.0mm Contour (紫) G71471 Abutment analog, ϕ 5.5mm, 5.0mm Contour (紫) G71431 Quick cap, ϕ 5.5mm, 5.0mm Contour (紫)</p>			
G71416	<p>Quick Packs, 6.0mm Contour, 0.5mm Cuff For 5.5/6.5mm Genesis Implants 內容物： G71450 Quick Abutment (含 screw-G71125) G71466 Temporary/Healing cap G71486 plastic sleeve, ϕ 5.5/6.5mm, Locking, 6.0mm Contour (紫) G71472 Abutment analog, ϕ 5.5/6.5mm, 6.0mm Contour (紫) G71432 Quick cap, ϕ 5.5/6.5mm, 6.0mm Contour (紫)</p>			
G71417	<p>Quick Packs, 6.0mm Contour, 1.5mm Cuff For 5.5/6.5mm Genesis Implants 內容物： G71451 Quick Abutment (含 screw-G71125) G71466 Temporary/Healing cap G71486 plastic sleeve, ϕ 5.5/6.5mm, Locking, 6.0mm Contour (紫) G71472 Abutment analog, ϕ 5.5/6.5mm, 6.0mm Contour (紫) G71432 Quick cap, ϕ 5.5/6.5mm, 6.0mm Contour (紫)</p>			
G71418	<p>Quick Packs, 6.0mm Contour, 2.5mm Cuff For 5.5/6.5mm Genesis Implants 內容物： G71452 Quick Abutment (含 screw-G71125) G71466 Temporary/Healing cap G71486 plastic sleeve, ϕ 5.5/6.5mm, Locking, 6.0mm Contour (紫) G71472 Abutment analog, ϕ 5.5/6.5mm, 6.0mm Contour (紫) G71432 Quick cap, ϕ 5.5/6.5mm, 6.0mm Contour (紫)</p>			
G71419	<p>Quick Packs, 6.0mm Contour, 4mm Cuff For 5.5/6.5mm Genesis Implants 內容物： G71453 Quick Abutment (含 screw-G71125) G71466 Temporary/Healing cap G71486 plastic sleeve, ϕ 5.5/6.5mm, Locking, 6.0mm Contour (紫) G71472 Abutment analog, ϕ 5.5/6.5mm, 6.0mm Contour (紫) G71432 Quick cap, ϕ 5.5/6.5mm, 6.0mm Contour (紫)</p>			
Genesis Quick Abutment (with screw-G71125)		<table border="1"> <tr> <td data-bbox="1142 1823 1353 1870">材質</td> <td data-bbox="1353 1823 1549 1870">滅菌方式</td> </tr> </table>	材質	滅菌方式
材質	滅菌方式			

 			
Quick Abutment(經 AnaTite 表面處理)		screw	
G71434	Quick Abutment, 5.0mm Contour, 1mm Cuff	For 3.8mm Genesis Implants	鈦合金
G71435	Quick Abutment, 5.0mm Contour, 2mm Cuff	For 3.8mm Genesis Implants	
G71436	Quick Abutment, 5.0mm Contour, 3mm Cuff	For 3.8mm Genesis Implants	
G71437	Quick Abutment, 5.0mm Contour, 4mm Cuff	For 3.8mm Genesis Implants	
G71438	Quick Abutment, 5.0mm Contour, 0.5mm Cuff	For 4.5mm Genesis Implants	
G71439	Quick Abutment, 5.0mm Contour, 1.5mm Cuff	For 4.5mm Genesis Implants	
G71440	Quick Abutment, 5.0mm Contour, 2.5mm Cuff	For 4.5mm Genesis Implants	
G71441	Quick Abutment, 5.0mm Contour, 4mm Cuff	For 4.5mm Genesis Implants	
G71442	Quick Abutment, 6.0mm Contour, 0.5mm Cuff	For 4.5mm Genesis Implants	
G71443	Quick Abutment, 6.0mm Contour, 1.5mm Cuff	For 4.5mm Genesis Implants	
G71444	Quick Abutment, 6.0mm Contour, 2.5mm Cuff	For 4.5mm Genesis Implants	
G71445	Quick Abutment, 6.0mm Contour, 4mm Cuff	For 4.5mm Genesis Implants	
G71446	Quick Abutment, 5.0mm Contour, 0.5mm Cuff	For 5.5mm Genesis Implants	
G71447	Quick Abutment, 5.0mm Contour, 1.5mm Cuff	For 5.5mm Genesis Implants	
G71448	Quick Abutment, 5.0mm Contour, 2.5mm Cuff	For 5.5mm Genesis Implants	
G71449	Quick Abutment, 5.0mm Contour, 4mm Cuff	For 5.5mm Genesis Implants	
G71450	Quick Abutment, 6.0mm Contour, 0.5mm Cuff	For 5.5/6.5mm Genesis Implants	
G71451	Quick Abutment, 6.0mm Contour, 1.5mm Cuff	For 5.5/6.5mm Genesis Implants	
G71452	Quick Abutment, 6.0mm Contour, 2.5mm Cuff	For 5.5/6.5mm Genesis Implants	
G71453	Quick Abutment, 6.0mm Contour, 4mm Cuff	For 5.5/6.5mm Genesis Implants	
Genesis Quick Abutment Temporary/Healing Cap			
			材質
Quick Abutment Temporary/Healing Cap			滅菌方式
G71462	Quick Abutment Temporary/Healing Cap, 5.0mm Contour	For 3.8mm Quick Abutment	PMMA
G71463	Quick Abutment Temporary/Healing Cap, 5.0mm Contour	For 4.5mm Quick Abutment	
G71464	Quick Abutment Temporary/Healing Cap, 6.0mm Contour	For 4.5mm Quick Abutment	
G71465	Quick Abutment Temporary/Healing Cap, 5.0mm Contour	For 5.5mm Quick Abutment	

G71466	Quick Abutment Temporary/Healing Cap, 6.0mm Contour For 5.5/6.5mm Quick Abutment		
Genesis Locator Abutment Assembly (Consists of Locator Abutment, Locator Denture Cap, Locator Replacement Male White, Locator White Block Out Spacer, Locator Replacement Male, LR, Pink)		材質	滅菌方式
 <p>Locator Abutment</p>  <p>Locator Denture Cap</p>  <p>Locator Replacement Male White</p>  <p>Locator Replacement Male, LR, Pink</p>  <p>Locator White Block Out Spacer</p>			
G71500	Locator Abutment Assembly, 1mm Cuff For 3.8mm Genesis Implants	1.Locator Abutment:鈦合金 2.Locator Denture Cap:鈦合金 3.Locator Replacement Male White: Nylon 4.Locator White Block Out Spacer: Silicon 5.Locator Replacement Male, LR, Pink: Nylon	Gamma
G71501	Locator Abutment Assembly, 2mm Cuff For 3.8mm Genesis Implants		
G71502	Locator Abutment Assembly, 3mm Cuff For 3.8mm Genesis Implants		
G71503	Locator Abutment Assembly, 4mm Cuff For 3.8mm Genesis Implants		
G71504	Locator Abutment Assembly, 5mm Cuff For 3.8mm Genesis Implants		
G71505	Locator Abutment Assembly, 6mm Cuff For 3.8mm Genesis Implants		
G71506	Locator Abutment Assembly, 1mm Cuff For 4.5mm Genesis Implants		
G71507	Locator Abutment Assembly, 2mm Cuff For 4.5mm Genesis Implants		
G71508	Locator Abutment Assembly, 3mm Cuff For 4.5mm Genesis Implants		
G71509	Locator Abutment Assembly, 4mm Cuff For 4.5mm Genesis Implants		
G71510	Locator Abutment Assembly, 5mm Cuff For 4.5mm Genesis Implants		
G71511	Locator Abutment Assembly, 6mm Cuff For 4.5mm Genesis Implants		
G71512	Locator Abutment Assembly, 1mm Cuff For 5.5mm Genesis Implants		
G71513	Locator Abutment Assembly, 2mm Cuff For 5.5mm Genesis Implants		
G71514	Locator Abutment Assembly, 3mm Cuff For 5.5mm Genesis Implants		
G71515	Locator Abutment Assembly, 4mm Cuff For 5.5mm Genesis Implants		
G71516	Locator Abutment Assembly, 5mm Cuff For 5.5mm Genesis Implants		
G71517	Locator Abutment Assembly, 6mm Cuff For 5.5mm Genesis Implants		
Genesis Snap Abutment Assembly (Consists of Abutment, Housing, Spacer)		材質	滅菌方式



Snap Abutment




Housing, Spacer

G71530	Snap Abutment Assembly, 2mm Cuff	For 3.8mm Genesis Implants	1.Snap Abutment: 鈦合金 2.Housing,: Stainless Steel 3.Spacer:Brass Gamma
G71531	Snap Abutment Assembly, 3mm Cuff	For 3.8mm Genesis Implants	
G71532	Snap Abutment Assembly, 4mm Cuff	For 3.8mm Genesis Implants	
G71534	Snap Abutment Assembly, 2mm Cuff	For 4.5mm Genesis Implants	
G71535	Snap Abutment Assembly, 3mm Cuff	For 4.5mm Genesis Implants	
G71536	Snap Abutment Assembly, 4mm Cuff	For 4.5mm Genesis Implants	
G71538	Snap Abutment Assembly, 2mm Cuff	For 5.5mm Genesis Implants	
G71539	Snap Abutment Assembly, 3mm Cuff	For 5.5mm Genesis Implants	




Occlusal plug

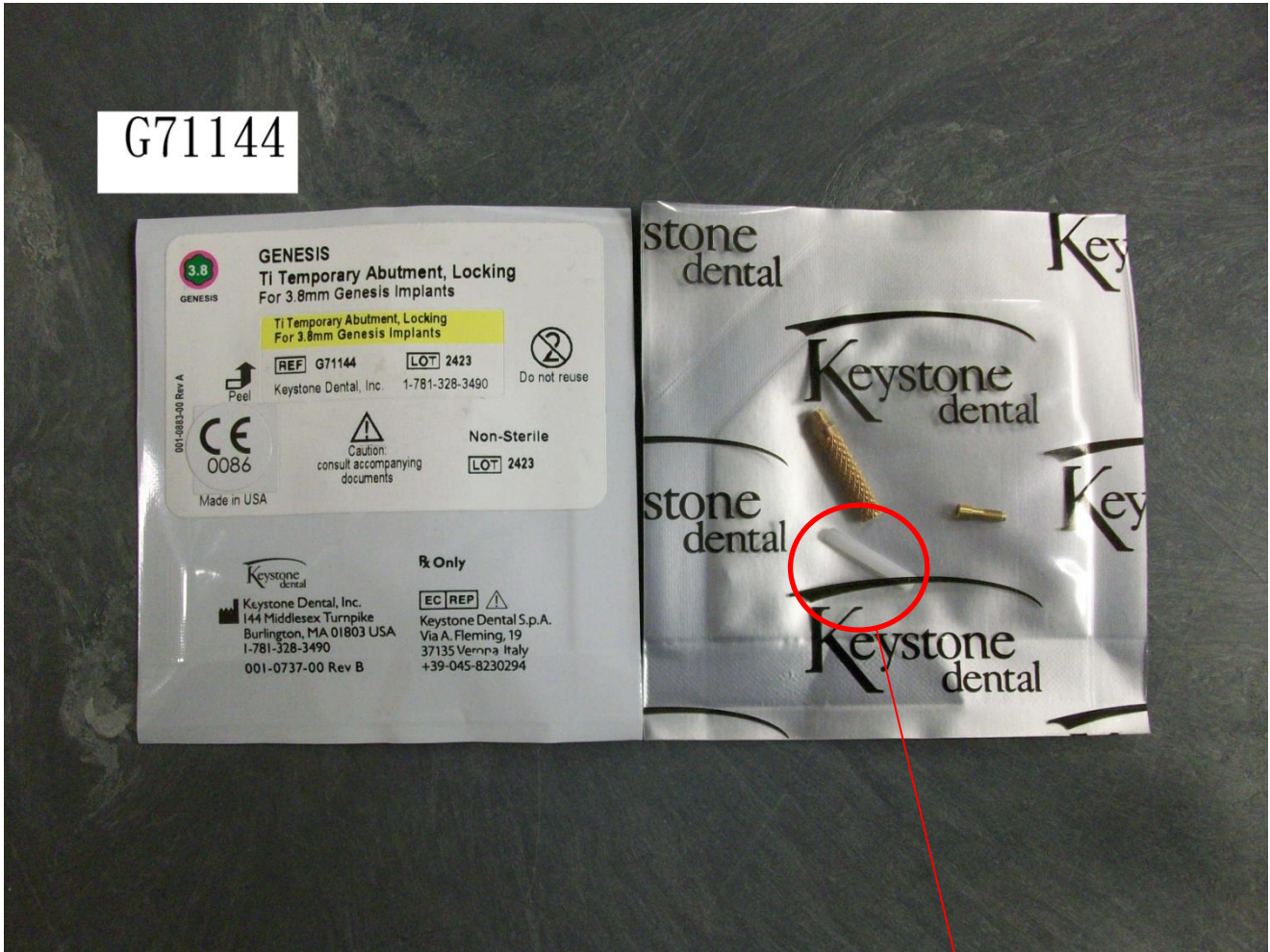

Occlusal plug

G71231

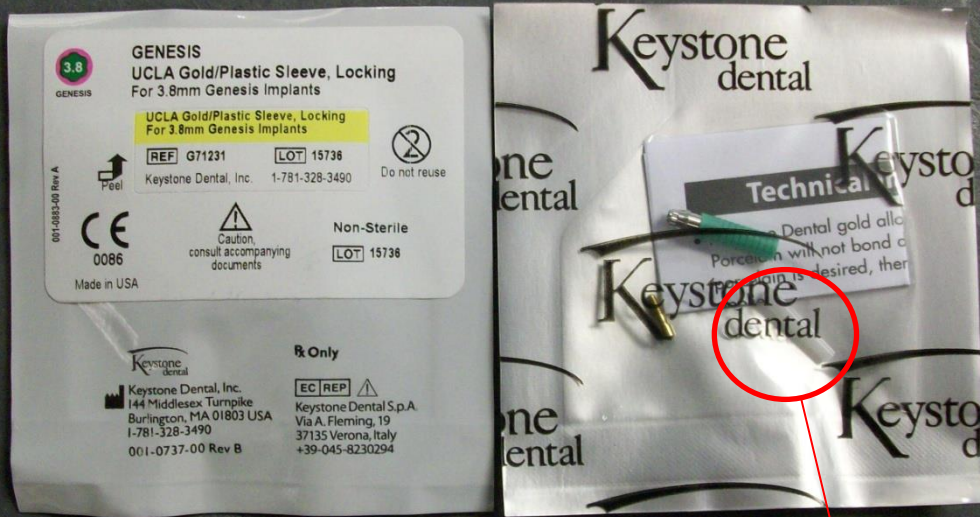

Occlusal plug