

“凱石” 牙科用植體基柱
“KEYSTONE” PrimaConnex Abutment

衛部醫器輸字第028813號

使用前請務必詳閱原廠之使用說明書並遵照指示使用。

■ 產品敘述及用途

本產品是與Keystone PrimaConnex牙科植體搭配使用，用於上下顎的單顆、多顆或覆蓋假牙膺復的支撐。本產品包括黏著固位、螺絲固位及覆蓋式固位。

黏著固位式膺復

Esthetic Contour Ti Abutment

可用於傳統牙冠和牙橋技術的單顆或多顆黏著固位式膺復。

具有直型和15° 兩種。

材質：鈦合金。

Quick-Abutment

可用於單顆或多顆的黏著式固位膺復。

材質：鈦合金。

Prepable Abutment

用於單顆或多顆的膺復。

材質：鈦合金。

螺絲固位式膺復

UCLA Abutment

用於單顆或多顆牙冠和牙橋膺復及球體附接固位覆蓋膺復。

材質：金合金，乙醛

Multi-Unit Abutment

用於多顆牙冠和牙橋的膺復，及球體附接覆蓋式膺復。不可用於單顆的膺復。

材質：鈦合金。

覆蓋式膺復

Locator Abutment

用於兩個以上牙科植體的附接覆蓋式膺復。Locator Abutment螺紋直接鎖入PrimaConnex牙科植體。

材質：鈦合金。

Snap Abutment

用於兩個以上牙科植體的附接覆蓋式膺復。

材質：鈦合金。

■ 禁忌症

- 一般外科手術必須注意的禁忌症。
- 可能禁忌症：慢性出血問題、心理障礙、骨代謝或結締組織疾病、皮質類固醇的治療、某些心臟與血管疾病、抽菸、糖尿病（未受控制）、化學藥物治療、慢性腎臟疾病、口腔衛生不良患者、夜間磨牙、酗酒。
- 暫時性禁忌症：全身性感染、局部口腔或呼吸道感染。

- 解剖或病理上的禁忌症：牙槽骨的寬度及高度少於1mm以至無法包覆植體。

■ 滅菌

本產品經確效之方法滅菌。請參考各產品包裝盒標籤的滅菌資訊；所有滅菌產品皆標示有“STERILE”。滅菌產品限單次使用並於保存期限前使用。勿使用外包裝有破損或包裝已被打開之滅菌產品。

■ 警告及注意事項

當植體或本產品負荷超過其功能可能發生過度的骨損失或植體或膺復設備的斷裂。生理和解剖的條件可能會對植體的功能有負面影響。當植入牙科植體時應考慮下列情況：

- 不好的骨質
- 不好的口腔衛生
- 醫療條件如血液疾病或未受控制的賀爾蒙分泌

本產品只能銷售給有證照的牙醫師或醫師。本產品不可與其他製造廠的牙科植體一起使用。

■ 患者的評估與選擇

成功的植牙手術必須有植牙醫師、膺復牙醫師和牙科實驗室技術人員的協調努力。對牙科植體長期功能適當的患者選擇是非常重要的。在做植牙手術時要先考慮下列情況：

- 患者病史
- 口腔衛生
- 患者的期望
- 一般牙科和產品的用途及禁忌症
- 植體位置的解剖標記
- 咬合間隙（牙槽脊和對應牙列間空間）
- 相應於植體直徑的牙橋寬度

■ 產品型號及規格

型號 產品敘述

Esthetic Contour Ti Abutment

45030K	Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 5.0mm Dia 1mm Cuff
45031K	Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 5.0mm Dia 3mm Cuff
45032K	15° Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 5.0mm Dia 1mm
45033K	15° Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 5.0mm Dia 3mm Cuff
45034K	Esthetic Contour Ti Abutment, RD for RD PrimaConnex Implants / 6.0mm Dia 1mm Cuff
45035K	Esthetic Contour Ti Abutment, RD for RD PrimaConnex Implants / 6.0mm Dia 3mm Cuff
45036K	15° Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 6.0mm Dia 1mm Cuff
45037K	15° Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 6.0mm Dia 3mm Cuff
45038K	Esthetic Contour Ti Abutment, WD for WD PrimaConnex Implants / 6.0mm Dia 1mm Cuff
45039K	Esthetic Contour Ti Abutment, WD for WD PrimaConnex Implants / 6.0mm Dia 3mm Cuff
45040K	15° Esthetic Contour Ti Abutment, WD for PrimaConnex Implants / 6.0mm Dia 1mm Cuff
45046K	Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 4.0mm Dia 1mm Cuff
45047K	Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 4.0mm Dia 2mm Cuff
45048K	15° Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 4.0mm Dia 1mm Cuff
45049K	15° Esthetic Contour Ti Abutment, SD for SD PrimaConnex Implants / 4.0mm Dia 2mm Cuff
45050K	Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 5.0mm Dia 0.5mm Cuff
45051K	Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 5.0mm Dia 1.5mm Cuff
45052K	15° Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 5.0mm Dia 0.5mm Cuff
45053K	15° Esthetic Contour Ti Abutment, RD for PrimaConnex Implants / 5.0mm Dia 1.5mm Cuff
45054K	Esthetic Contour Ti Abutment, WD for PrimaConnex Implants / 5.0mm Dia 0.5mm Cuff

- 45055K Esthetic Contour Ti Abutment, WD for PrimaConnex Implants / 5.0mm Dia 1.5mm Cuff
- 45056K 15° Esthetic Contour Ti Abutment, WD for PrimaConnex Implants / 5.0mm Dia 0.5mm Cuff
- 45057K 15° Esthetic Contour Ti Abutment, WD for PrimaConnex Implants / 5.0mm Dia 1.5mm Cuff

Quick-Abutment

- 45420K Quick-Abutment, SD For SD PrimaConnex Implants / 1mm Cuff 5.0mm Diameter
- 45421K Quick-Abutment, SD For SD PrimaConnex Implants / 2mm Cuff 5.0mm Diameter
- 45422K Quick-Abutment, SD For SD PrimaConnex Implants / 3mm Cuff 5.0mm Diameter
- 45423K Quick-Abutment, SD For SD PrimaConnex Implants / 4mm Cuff 5.0mm Diameter
- 45424K Quick-Abutment, RD For RD PrimaConnex Implants / 0.5mm Cuff 6.0mm Diameter
- 45425K Quick-Abutment, RD For RD PrimaConnex Implants / 1.5mm Cuff 6.0mm Diameter
- 45426K Quick-Abutment, RD For RD PrimaConnex Implants / 2.5mm Cuff 6.0mm Diameter
- 45427K Quick-Abutment, RD For RD PrimaConnex Implants / 4.0mm Cuff 6.0mm Diameter
- 45428K Quick-Abutment, WD For WD PrimaConnex Implants / 0.5mm Cuff 6.0mm Diameter
- 45429K Quick-Abutment, WD For WD PrimaConnex Implants / 1.5mm Cuff 6.0mm Diameter
- 45430K Quick-Abutment, WD For WD PrimaConnex Implants / 2.5mm Cuff 6.0mm Diameter
- 45431K Quick-Abutment, WD For WD PrimaConnex Implants / 4.0mm Cuff 6.0mm Diameter
- 45435K Quick-Abutment, RD For RD PrimaConnex Implants / 0.5mm Cuff 5.0mm Diameter
- 45436K Quick-Abutment, RD For RD PrimaConnex Implants / 1.5mm Cuff 5.0mm Diameter
- 45437K Quick-Abutment, RD For RD PrimaConnex Implants / 2.5mm Cuff 5.0mm Diameter
- 45438K Quick-Abutment, RD For RD PrimaConnex Implants / 4.0 mm Cuff 5.0mm Diameter
- 45439K Quick-Abutment, WD For WD PrimaConnex Implants / 0.5mm Cuff 5.0mm Diameter
- 45440K Quick-Abutment, WD For WD PrimaConnex Implants / 1.5mm Cuff 5.0mm Diameter
- 45441K Quick-Abutment, WD For WD PrimaConnex Implants / 2.5mm Cuff 5.0mm Diameter
- 45442K Quick-Abutment, WD For WD PrimaConnex Implants / 4.0mm Cuff 5.0mm Diameter

Prepable Abutment

- 45042K Prepable Abutment, SD For SD PrimaConnex Implants
- 45043K Prepable Abutment, RD For RD PrimaConnex Implants
- 45044K Prepable Abutment, WD For WD PrimaConnex Implants

UCLA Abutment

- 45100K UCLA Gold/Plastic Sleeve, SD For SD PrimaConnex Implants / Locking
- 45101K UCLA Gold/Plastic Sleeve, SD For SD PrimaConnex Implants / Non-Locking
- 45102K UCLA Gold/Plastic Sleeve, RD For RD PrimaConnex Implants / Locking
- 45103K UCLA Gold/Plastic Sleeve, RD For RD PrimaConnex Implants / Non-Locking
- 45104K UCLA Gold/Plastic Sleeve, WD For WD PrimaConnex Implants / Locking
- 45105K UCLA Gold/Plastic Sleeve, WD For WD PrimaConnex Implants / Non-Locking

Multi-Unit Abutment

- 45312K Multi-Unit Abutment, Straight, 1mm Cuff 3.5/3.8mm TiLobe Implants
- 45313K Multi-Unit Abutment, Straight, 2mm Cuff 3.5/3.8mm TiLobe Implants
- 45314K Multi-Unit Abutment, Straight, 3mm Cuff 3.5/3.8mm TiLobe Implants
- 45315K Multi-Unit Abutment, Straight, 1mm Cuff 4.1/4.5 mm TiLobe Implants
- 45316K Multi-Unit Abutment, Straight, 2mm Cuff 4.1/4.5mm TiLobe Implants
- 45317K Multi-Unit Abutment, Straight, 3mm Cuff 4.1/4.5mm TiLobe Implants
- 45318K Multi-Unit Abutment, Straight, 4mm Cuff 4.1/4.5mm TiLobe Implants
- 45319K Multi-Unit Abutment, Straight, 1mm Cuff 5.0/5.5/6.5mm TiLobe Implants
- 45320K Multi-Unit Abutment, Straight, 2mm Cuff 5.0/5.5/6.5mm TiLobe Implants
- 45321K Multi-Unit Abutment, Angled, 17°, 3mm Cuff 3.5/3.8mm TiLobe Implants
- 45322K Multi-Unit Abutment, Angled, 17°, 4mm Cuff 3.5/3.8mm TiLobe Implants
- 45323K Multi-Unit Abutment, Angled, 17°, 3mm Cuff 4.1/4.5mm TiLobe Implants
- 45324K Multi-Unit Abutment, Angled, 17°, 4mm Cuff 4.1/4.5mm TiLobe Implants

45325K Multi-Unit Abutment, Angled, 30°, 3mm Cuff 4.1/4.5mm TiLobe Implants
45326K Multi-Unit Abutment, Angled, 30°, 4mm Cuff 4.1/4.5mm TiLobe Implants

Zest Locator Abutment

45062K Zest Locator Abutment Assembly, SD For SD PrimaConnex Implants / 1mm Cuff
45063K Zest Locator Abutment Assembly, SD For SD PrimaConnex Implants / 5mm Cuff
45064K Zest Locator Abutment Assembly, SD For SD PrimaConnex Implants / 6mm Cuff
45065K Zest Locator Abutment Assembly, RD For RD PrimaConnex Implants / 1mm Cuff
45066K Zest Locator Abutment Assembly, RD For RD PrimaConnex Implants / 5mm Cuff
45067K Zest Locator Abutment Assembly, RD For RD PrimaConnex Implants / 6mm Cuff
45068K Zest Locator Abutment Assembly, WD For WD PrimaConnex Implants / 1mm Cuff
45069K Zest Locator Abutment Assembly, WD For WD PrimaConnex Implants / 4mm Cuff
45070K Zest Locator Abutment Assembly, WD For WD PrimaConnex Implants / 5mm Cuff
45071K Zest Locator Abutment Assembly, WD For WD PrimaConnex Implants / 6mm Cuff

Snap Abutment

45250K Snap Abutment Assembly, SD For SD PrimaConnex Implants / 2mm Cuff
45251K Snap Abutment Assembly, SD For SD PrimaConnex Implants / 3mm Cuff
45252K Snap Abutment Assembly, SD For SD PrimaConnex Implants / 4mm Cuff
45253K Snap Abutment Assembly, RD For RD PrimaConnex Implants / 2mm Cuff
45254K Snap Abutment Assembly, RD For RD PrimaConnex Implants / 3mm Cuff
45255K Snap Abutment Assembly, RD For RD PrimaConnex Implants / 4mm Cuff
45256K Snap Abutment Assembly, WD For WD PrimaConnex Implants / 2mm Cuff
45257K Snap Abutment Assembly, WD For WD PrimaConnex Implants / 3mm Cuff

Abutment Screw

45060K Abutment Screw, SD/RD/WD for PrimaConnex Abutments

使用器械

Hader Bar and Clip System

ISS185HK Hader Castable Bar & Clip System 100mm Bar, 12 Clips, & Clip Tool
ISS185BK Hader Castable Bar
ISS185HPK Hader Processing Clips, 6 Pack
ISS185HRK Hader Final Clips, 6 Pack
ISS185TK Hader Clip Insertion Tool
ISS526HK Hader Clip Metal Housings, 6 Pack

製造廠名稱：KEYSTONE DENTAL, INC.

製造廠地址：5 Holland Building 209 Irvine, CA 92618 USA

藥商名稱：杏昌生技股份有限公司

藥商地址：台北市中正區仁愛路二段47號5樓



Esthetic Contour Ti Abutment (45030K)



Quick-Abutment (45420K)



Prepable Abutment (45042K)



UCLA Abutment (45100K)



Multi-Unit Abutment (45312K)



Zest Locator Abutment (45062K)



Snap Abutment (45250K)



Abutment Screw (45060K)